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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The ID MOUV leisure chair is a device designed to provide mobility in a seated position for disabled or elderly people.. The chair is intended for indoor and outdoor use over short distances and folds up for easy storage or transport.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 100 Kg.
- Make sure that all parts (wheels, cables, handles, knobs, joystick, etc...) are secure and in good working order.
- Do not use the wheelchair on stairs or escalators.
- Only use accessories and parts supplied by the manufacturer.
- Do not make any adjustments to the wheelchair while it is in use.
- Check that the joystick works properly before use.
- All wheels must always be in contact with the ground.
- Do not use the walker on a wet, icy, or snowy slope.
- Do not stand on the footrest as this could cause the wheelchair and the user to fall.
- When the wheelchair is used on sloping ground: the user must fasten the seat belt.
- If any parts are damaged, broken or worn, contact the dealer.
- Ensure that the user has the ability to use the product by running a course

PRODUCT DESCRIPTION

1	Handle tubing
2	Joystick
3	Seat
4	Front frame
5	Battery box
6	Footrest
7	Front wheel Ø 15 cm
8	Rear wheel Ø 20 cm
9	Anti-tip wheel
10	Brake
11	Rear frame
12	Safety belt
13	Backrest
14	Armrest
15	Locking knob

MOUNTING & USE

MOUNTING :

- **Wheelchair opened :** Hold the backrest in one hand and the front frame in one hand and unfold at the same time (as shown in Figures 1 and 2) to the four-wheel landing and press the backrest to make the folding switch iron hook on the back frame bushin and the wheelchair opens. The wheelchair is unfolded (3).

(1)


(2)

(3)

- **Wheelchair folding :** Turn the pedals and armrests up first, then pull the folding switch frame together and press down on the backrest, the wheelchair can be easily folded.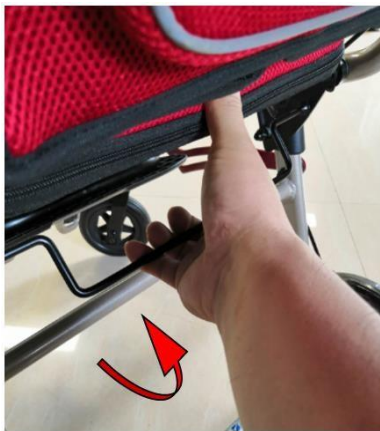

- **Handle operation :** Release the handle of the quick release lever to adjust the height of the handle or remove the handle, and lock it to fix it.

- **Remote control adjustment operation** : Release the handle of the quick release lever to adjust or remove the remote control. Locking is fixed.


- **Anti-tip wheel installation** : Install the anti-tip wheel and fix it with the bolt.

- **Remote control wiring** : The remote control is installed on the right by default. Plug the control wire connector on the controller with the corresponding connector on the control box and tighten the connection nut.

When mating the joints, pay attention to the matching of the notches. The black arrows on the left and right joints must correspond. Do not insert them forcibly to prevent damage to the joints.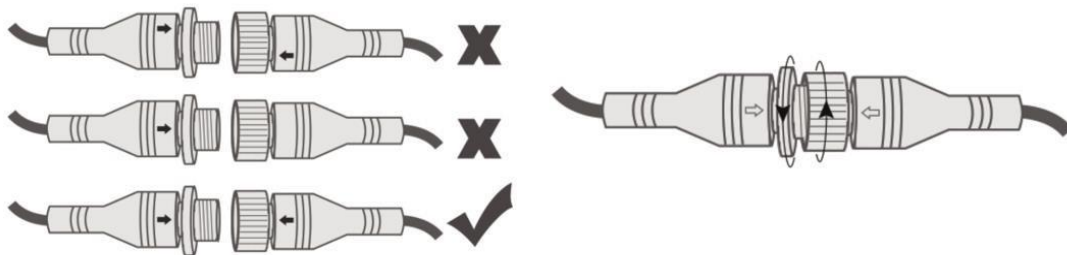

USE :

- ❖ After the operator is seated firmly, turn on the power switch **1** of the controller, and then promptly turn on the wheelchair. The system sets a safe starting strategy. Please release the joystick when starting, otherwise it will prompt the system to fail and fail to start.
- ❖ Wheelchair driving speed is adjusted by the speed adjustment button **3** or **7** (+ or -) on the controller. There are five gears. It is advisable to use low speed when starting. Real-time speed can be displayed during driving, and gears cannot be changed during driving. First-time users recommend low speed Driving; to avoid accidents due to unskilled operation.
- ❖ Slowly start by pulling the handle **4**, the rocker handle will move forward to the wheelchair, the rocker handle will turn left to the wheelchair, the rocker handle will turn right to the wheelchair, and the wheelchair will move backward. Allows wheelchairs to travel freely.
- ❖ If the whistle alarm is required, press the button marked with the horn symbol on the controller **6**.
- ❖ When the battery is fully charged **5**, it displays 5 power levels. When the power is insufficient, the power level is 1 or the power icon flashes to indicate that the current remaining power is insufficient. Please charge in time to keep the power sufficient
- ❖ To turn off, press and hold the power switch **1** for a long time and turn off the wheelchair after a prompt.

Remote control terminal function description :

1	Power switch
2	Gear
3	Speed limit reduction
4	Joystick
5	Battery indicator
6	Horn
7	Speed limit increase
8	Joystick

The remote control is switched on when you hear a beep. To turn off the remote control, press and hold the power button for 3 seconds. The remote control is switched off once you hear a beep.

Electronic brake :

This electric wheelchair adopts an electronic brake system. It can be stopped immediately when the user releases his hand. Do not turn off the power when the user is going up or down the road. If you want to park or get off the car, please pull the brake first. Wheelchairs slide automatically ; press this material to ensure safe use. Users are advised to turn off the controller without using a wheelchair to reduce power consumption.

Low battery smart brake :

The ID Mouv electric wheelchair can be operated in electric or manual mode. The electric mode is automatically activated when the wheelchair is switched on. To switch to manual mode, switch off the joystick.

During your journey, if the battery of the wheelchair is less than 10%, you must stop the journey and recharge the battery. Switch to manual mode to finish your journey.

When using the wheelchair on rough roads, we advise you to keep the manual mode to ensure your safety.

Charging :

For safety reasons, always use the charger supplied with the ID Mouv wheelchair. Plug the charger's output plug into the charging socket on the back of the joystick.

Connect the charger's power plug to a wall socket suitable for the voltage and frequency of the charger.

When the charging indicator on the charger turns green, it means that the power is sufficient.

The battery capacity is 6Ah. The charging time is about 4-5h.

When getting up from the chair, please do not lean on the footrest to avoid tipping the chair. When getting in or out of the chair, always lock the brake and raise the footrest to prevent falls.


CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the wheelchair regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Do not swap the left and right motor connectors. In order not to cause an incorrect connection (see diagram).
- Check the joystick attachment on the right or left and the condition of the wires from the wheel casings to the joystick (once a week)
- Check the freewheel function and engagement (before each use)
- Check the battery (once a month)
- Check folding and unfolding, product clicked in unfolded position (once a month)
- Check product speed (before each use)
- If there is a malfunction, do not repair the wheelchair yourself. Consult an authorized dealer.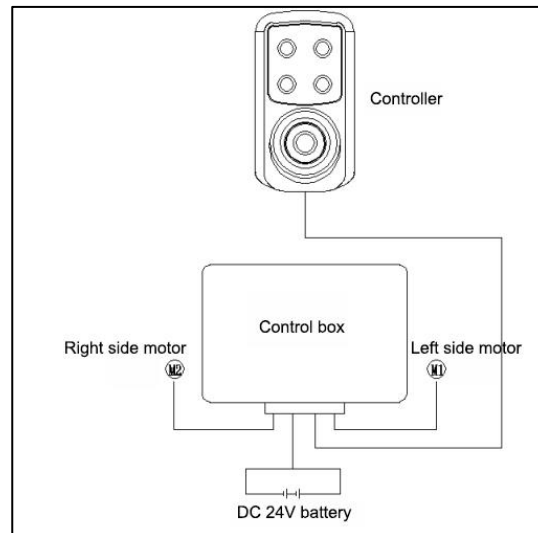

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Larg. 58 x Prof. 93 x Haut. 97 cm
--------------------	-----------------------------------

Seat dimensions	Larg. 42 x Prof. 36 x Haut. 53 cm
Backrest height	40 cm
Folded width	34 cm
Motors	2x DC24V 250W
Autonomy	15 à 20 km
Battery capacity	Lithium 6 aH
Maximum speed	6 km/h
Minimum turning radius	≤ 1200 mm
Braking performance	≤ 1,5 m
Weigth	18 kg
Maximum supported weight	100 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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