# Otto Bock®

QUALITY FOR LIFE

## 28U11

(D)	<b>Otto Bock Unterschenkelorthese Walk On®</b> .....	3
(GB)	<b>Otto Bock Walk On® AFO orthosis</b> .....	4
(F)	<b>Otto Bock orthèse tibiale Walk On®</b> .....	5
(I)	<b>Ortesi tibiale Otto Bock Walk On®</b> .....	7
(E)	<b>Walk On® Órtesis tibial de Otto Bock</b> .....	8
(P)	<b>Walk On® Ortótese tibial da Otto Bock</b> .....	9
(S)	<b>Otto Bock Walk On® AFO ortos</b> .....	11
(NL)	<b>Otto Bock Walk On® AFO</b> .....	12
(DK)	<b>Otto Bock Walk On® AFO ortose</b> .....	13
(FIN)	<b>Otto Bock Walk On® AFO ortoosi</b> .....	15
(PL)	<b>Otto Bock orteza podudzia typu Walk On®</b> .....	16
(H)	<b>Otto Bock Walk On® lábászár ortézis</b> .....	17
(CZ)	<b>Bércová ortéza Otto Bock Walk On®</b> .....	19
(GR)	<b>Ποδοκνημικός νάρθηκας Walk On® της Otto Bock</b> .....	20
(RUS)	<b>Голеностопный ортез «Walk On®»</b> .....	22

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## **1 Application**

English

The 28U11 Walk On® Ankle Foot orthosis (AFO), pre-fabricated by Otto Bock Scandinavia AB, is **solely** intended for the orthotic fitting of the lower extremities.

## **2 Indications and Function**

The orthosis is designed to help those patients who need foot lever function (for example in cases of peroneal paralysis or after an apoplexy stroke) to achieve a more natural gait pattern.

The Walk On® lifts the foot during swing phase and limits plantar flexion.

Note: The Walk On® is a dynamic ankle foot orthosis and must not be used for the prevention of contractures, etc. The indications must be provided by the physician.

## **3 Selection of Orthosis Size**

Article No.	Side	Shoe size
28U11=L small	left	36 - 39
28U11=R small	right	36 - 39
28U11=L medium	left	39 - 42
28U11=R medium	right	39 - 42
28U11=L large	left	42 - 45
28U11=R large	right	42 - 45
28U11=L X-large	left	45 - 48
28U11=R X-large	right	45 - 48

## **4 Construction**

Thanks to its construction which includes a dorsal spring and a rigid connection to the springy sole, the orthosis provides a smooth natural gait on uneven ground.

### **Attention**

The orthosis is made of pre-impregnated carbon fiber material and cannot be thermoformed. The foot sole and the connection elements must be free from holes **as these would interrupt the fibers and weaken the component.**

## 5 Trial Fitting

Sole and calf shell can be ground to make adjustments. When adjusting the sole width, grind the lateral side no more than is required so that the medial part of the spring does not press against the foot. If the sole requires more reduction in width, then grind bilaterally (Fig. 1). Please note that when adjusting the length the connection brace between sole and dorsal spring must not get too close to the medial side of the ankle. Shortening the heel section brings the connection brace into a dorsal position, thus preventing direct contact with the ankle (Fig. 2 & 3).

An insole can be pasted onto the carbon sole to improve the foot bedding.

After adjustment, the edges should be sanded smooth to avoid any discomfort or unnecessary wear on the shoe.

### **Note!**

The user of the above-mentioned product **must be informed of the possible risks** that exceptional situations might present. For example, jumping down from great height may expose the spring to severe overload and cause it to break; or the orthosis could prevent the wearer from braking quickly and suddenly while driving a car.

## 6 Liability

The manufacturer's warranty applies only if the product has been used under the conditions and for the purposes described. The manufacturer recommends that the product be used and maintained according to the instructions for use.

## 7 CE conformity

This product meets the requirements of the 93/42/EWG guidelines for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Otto Bock with sole responsibility according to appendix VII of the guidelines.

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**Otto Bock HealthCare GmbH**