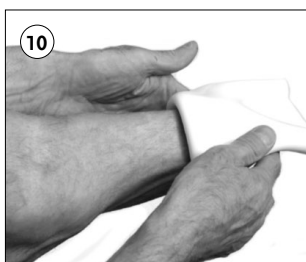
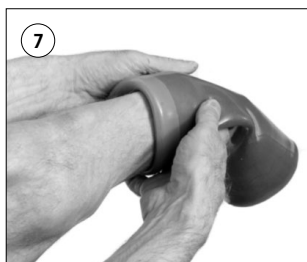
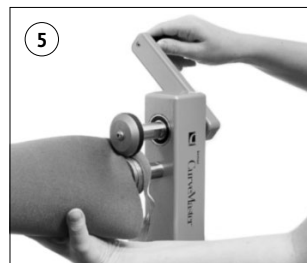
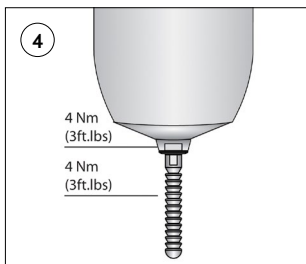
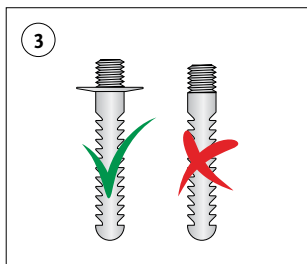
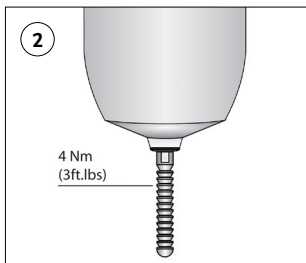
Instructions for Use

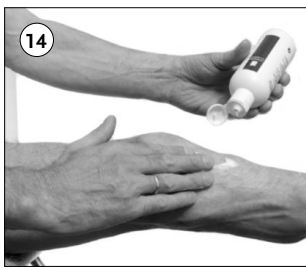
---

ICEROSS<sup>®</sup>  
CUSHION & LOCKING

		3	
DE		Gebrauchsanweisung	9
FR		Notice d'utilisation	13
ES		Instrucciones para el uso	17
IT		Istruzioni per l'uso	21
DA		Brugsanvisning	25
SV		Bruksanvisning	29
EL		Οδηγίες χρήσης	33
FI		Käyttöohjeet	38
NL		Gebruiksaanwijzing	42
PT		Instruções de Utilização	46
PL		Instrukcja użytkowania	50
CS		Návod k použití	54
TR		Kullanım Talimatları	58
RU		Инструкция по использованию	62
JA		取扱説明書	67
ZH		中文说明书	71
KO		사용 설명서	74

## INTRODUCTION

Thank you for choosing a high quality silicone liner from Össur. We can assure you that you have made the right choice. Please carefully review and follow these instructions before fitting a patient with the liner or accessory products. Remember that informing the patient concerning proper handling and care of the liner, as set forth in these instructions, is important to achieve patient comfort and maximum product life.

## PRODUCT SELECTION AND FITTING

### **Liner sizing (Figure 1)**

To determine the correct Iceross size:

1. Measure the circumference of the residual limb at 4 cm from the distal end.
2. Choose the Iceross size, which corresponds to the measurement or which, is immediately below the indicated measurement.
3. Never choose a liner size, which exceeds the measurement, or round up.

**Example:** If the residual limb measures 24,5 cm at the indicated location, choose Iceross size 23,5 **NOT** size 25.

Selecting the correct size is very important. If the liner is too tight, this may cause pistoning and/or numbness and blistering. If the liner is too loose, increased perspiration and/or movement of the limb inside the liner may occur. Perspiration and movement can lead to blisters and rashes. If any of the above symptoms or any other indication of improper fit are noted, the user should be instructed to contact his/her physician or prosthetist immediately.

**Note:** On transtibial applications the stabilizing matrix should cover at least 4 cm of bony structures. Do not allow the matrix to extend higher than the fibula head. Similar principles apply to transradial applications, and the matrix should not extend higher than the medial condyles.

### **Distal attachments**

Your Iceross locking liner will have either fixed or interchangeable distal attachment, note that cushion liners have no distal attachments.

Cushion liners needs to be used with sleeve that holds vacuum.

### **Fixed Distal Attachments (Figure 2)**

When using an Iceross liner with a fixed distal attachment and an attachment pin, caution must be taken to properly tighten the attachment pin to the fixed distal attachment. Always apply Loctite 410/411 or equivalent compound to the threads of the Össur attachment pin (used with Icelock) and torque to 4 Nm (3ft.lbs). Össur does not recommend the use of pins without a flange with ICEROSS® locking liners (**Figure 3**). If you choose to use a locking pin without a flange please take note of the following actions:

Fasten the pin with maximum torque of 1,5 Nm (1ft. lbs) and apply Loctite 410/411 or equivalent compound to the threads of the pin.

### ***Interchangeable Distal Attachments (Figure 4)***

To use an Iceross liner with an interchangeable distal attachment be sure to tighten the distal attachment to the liner at 4 Nm (3ft. lbs) using a torque wrench. To fasten the attachment pin to the distal attachment, apply Loctite 410/411 or equivalent compound to the threads of the Össur attachment pin (used with Icelock) and torque to 4 Nm (3ft.lbs). If excessive movement or product wear is detected between the attachment pin and the distal attachment (fixed or interchangeable) at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her physician or prosthetist.

### ***Trimming an Iceross Liner (Figure 5)***

Iceross liners may be trimmed for user preference and to increase range of motion. The Iceross CurveMaster is specifically designed for this function and to produce a smooth, rounded edge, which will minimize the risk of skin irritation and/or tears of the liner. Do not cut the liner below the level of the matrix or below the proximal trim-lines of the socket. Excessive trimming may reduce the natural suction between the liner and limb and/or compromise suspension.

### ***Donning an Iceross Liner***

When donning the liner, turn the liner inside out and grip it as shown in the illustration (**Figure 6**). If you are donning an Iceross Activa liner make sure that the TibiaGuard is aligned along the anterior side of the tibia.

**Note:** Be very careful that the inside of the liner is clean, dry and free from any foreign objects that can cause skin irritation.

After exposing as much of the distal end of the liner as possible, position it against the residual limb and with light compression roll upward onto the limb. Check that no air pockets are present (**Figure 7**).

Roll the liner all the way up the limb, taking care not to damage it with fingernails. Do not tug or pull (**Figure 8**).

When using a liner without a textile cover Iceross Original, Iceross Clean and Simple lubricant spray can be used to facilitate donning. Spray the lubricant onto the OUTER surface of the liner, invert the liner and roll on. Wipe off any excess lubricant (**Figure 9**).

If talcum powder is used rather than Iceross Clean and Simple lubricant, be sure to lightly dust the OUTSIDE of the liner only.

### ***Iceross Distal Cup***

Iceross Distal Cups provide enhanced comfort for conical limbs with poor soft tissue coverage. They can also be used as a fitting tool in conjunction with the Iceross Original liners to ensure total contact between the silicone and residual limb. The DERMOGEL silicone in the Iceross Distal Cup conforms to the limb, thus minimizing pressure over the bony prominence during gait.

To correctly fit a Distal Cup:

1. Measure the circumference of the residual limb at 4 cm from the distal end (**Figure 1**).
2. Choose the Distal Cup two or three sizes smaller than the measurement
3. Roll the Distal Cup directly over the limb (**Figure 10**).

4. When using an Iceross liner in combination with a Distal Cup, the liner measurement (4 cm from the distal end) should be made with the Distal Cup in place (i.e. OVER the Distal Cup) (**Figure 11**).
5. Choose the correct Iceross liner size based on the measurement obtained with the Distal Cup in place

### **Iceross Pads (Figure 12)**

Iceross Pads can be used to fill in any problem areas (i. e. voids or scars) to ensure better fit and to minimize trapped air. Iceross Pads may also be used to distribute body pressure and to protect the skin. In some cases, custom-formed pads may be required for irregular shapes or problem areas.

### **Liner Care (Figure 13)**

Proper product care is important. The liner needs to be washed inside and out every day, following use. Remove the liner, turn the liner inside out and wash it with pH balanced, 100% fragrance- and dye-free soap. The liner can also be machine washed (40 °C/Hot) with a mild detergent. Fabric softeners or bleaches or other products or cleaning solutions may damage the liner and should **NOT** be used. After cleaning, the patient should rinse the liner thoroughly with water and wipe both sides dry with a lint-free cloth.

An Iceross liner can be used directly after washing. It does not need to dry overnight. Always return the liner to its normal position with the distal attachment facing out as soon as possible after cleaning.

Iceross liners should always be checked for damage or wear.

Any damage may weaken the effectiveness of the liner and should be reported to the user's prosthetist immediately. Misuse may result in potential loss of suspension.

**Note:** Care must be taken by both the prosthetist and the patient not to expose a liner to glass or carbon fibers or other foreign particles. Such substances can become embedded in the silicone causing aggravation of the skin. Washing alone may not be sufficient to eliminate the problem and the patient should be instructed to return the liner to the prosthetist if the liner is inadvertently exposed to any foreign substances or chemicals.

DermoSil and DermoGel silicone contain Aloe Vera extract. This acts as a natural moisturizer and is known for its active healing and soothing effects on skin. Initially, the inner surface of the liner should be moist to the touch. For these reasons, the liner should not be left inside out for extended period or exposed to excessive heat or sunlight.

### **Skin Care (Figure 14)**

Daily cleaning of the residual limb is also essential. We recommend use of a mild liquid, pH balanced, 100% fragrance and dye-free soap.

If dry skin is noted, apply a pH balanced, 100% fragrance- and dye-free lotion to nourish and soften the skin. Do **NOT** apply lotion to the residual limb immediately prior to donning the liner.

**Note:** Many common household or bath products, including soaps, deodorants, perfumes, aerosol or alcohol sprays or abrasive cleaners may cause or contribute to skin irritation and the patient should be so advised.

## PRODUCT WARRANTY

Össur offers a written warranty on the following products for the indicated period from the invoice date. Said warranty shall be in lieu of all others, whether express or implied by law.

Iceross Activa	9 months
Iceross Original w/o cover	6 months
Iceross SportK4	9 months
Iceross Synergy	9 months
Iceross Transfemoral	6 months
Iceross Original with cover	6 months
Iceross Comfort 3 mm	6 months
Iceross Comfort 6 mm (plus)	6 months
Iceross Dermo	6 months
Iceross Junior	6 months

For further information, please contact Össur or your local Össur distributor. You can also visit the Össur web page at [www.ossur.com](http://www.ossur.com).

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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