




Instructions for Use

ICEROSS SEAL-IN[®] X5

For TT/TF users

		3
EN	INSTRUCTIONS FOR USE	5
DE	GEBRAUCHSANWEISUNG	10
FR	NOTICE D'UTILISATION	15
ES	INSTRUCCIONES PARA EL USO	20
IT	ISTRUZIONI PER L'USO	25
DA	BRUGSANVISNING	30
SV	BRUKSANVISNING	35
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	39
FI	KÄYTTÖOHJEET	44
NL	GEBRUIKSAANWIJZING	49
PT	INSTRUÇÕES DE UTILIZAÇÃO	54
PL	INSTRUKCJA UŻYTKOWANIA	59
CS	NÁVOD K POUŽITÍ	64
TR	KULLANIM TALIMATLARI	69
RU	ИНСТРУКЦИЯ ПО ИСПОЛЬЗОВАНИЮ	74
JA	取扱説明書	79
ZH	中文说明书	83
KO	사용 설명서	87

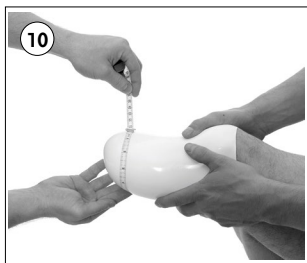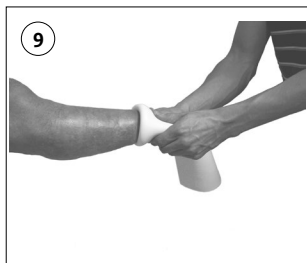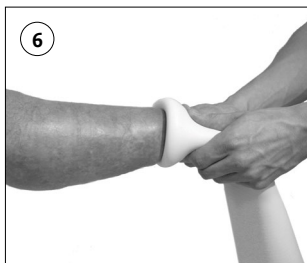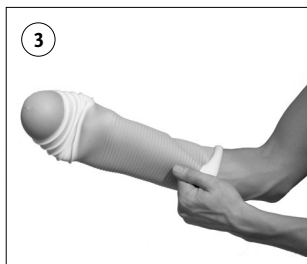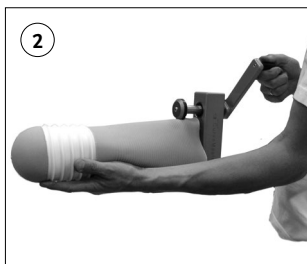11

12

13

ENGLISH

INTRODUCTION

Thank you for choosing a high quality silicone liner from Össur. We can assure you that you have made the right choice. Please carefully review and follow these instructions before fitting a patient with the liner or accessory products. Remember that informing the patient concerning proper handling and care of the liner, as set forth in these instructions, is important to achieve patient comfort and maximum product life.

CONTRAINDICATIONS FOR USE

Contraindications are:

1. Unstable limb volume.
2. Extremely conical limbs (*See ICEROSS® DISTAL CUP*).
3. Very short residual limbs.

Minimum recommended length:

Transtibial: To assess the minimum recommended length, measure the residual limb length from the mid patella tendon to the distal end, with knee in full extension.

Transfemoral: Measure the residual limb length with the tissue hanging down, from the perineum to the distal end.

- Compare the measurement to the table below.
- Low trimlines of the hard socket can affect suspension and should be assessed on individual basis.

Liner Sizes	Minimum Length
18–23	11.0 cm
25–30	12.5 cm
32–40	13.5 cm
45–50	15.5 cm
55	16.5 cm

NOTE: Low posterior trim lines of the hard socket can affect suspension and should be assessed on individual basis.

LINER SIZING (Figure 1)

To determine the correct Iceross size:

1. Measure the circumference of the residual limb at 4 cm from the distal end with the tissue hanging down.
2. Choose the Iceross size, which corresponds to the measurement or which, is immediately below the indicated measurement.
3. **DO NOT** choose a liner size, which exceeds the measurement, or round up.

EXAMPLE: If the residual limb measures 31 cm at the indicated location, choose size 30 **NOT** size 32.

Selecting the correct size is very important. If the liner is too tight, this may cause pistoning and/or numbness and blistering. A Seal-In liner that is too tight will also produce excessive ring tension to the residual limb from the seals. If the liner is too loose, increased perspiration and/or movement of the limb inside the liner may occur. Perspiration and movement can lead to blisters and rashes. If any of the above symptoms or any other indication of improper fit are noted, the user should be

instructed to contact his/her physician or prosthetist immediately.

TRIMMING AN ICEROSS LINER (Figure 2)

Iceross liners may be trimmed for user preference and to increase range of motion. The Iceross CurveMaster is specifically designed for this function and to produce a smooth, rounded edge, which will minimize the risk of skin irritation and/or tears of the liner.

Do not cut the liner below or through any of the Seals or below the proximal trim-lines of the socket. Excessive trimming may reduce the natural suction between the liner and limb and/or compromise suspension.

DONNING THE ICEROSS SEAL-IN® X5

To don the liner use the Clean & Simple Lubricant Spray (*See ICEROSS® CLEAN & SIMPLE LUBRICANT SPRAY*).

Before donning a new liner clean it with lukewarm water, and dry it with lint free cloth as the natural oils in the silicone make it difficult to grip.

1. Turn the liner inside out by gripping the distal end from the inside as shown in the illustration. **(Figure 3)**.
2. Grip the top of the liner and slide it over the hand until the inner surface is fully exposed. **(Figure 4)**.
3. Spray lubricant into the inverted liner for easier roll on. **(Figure 5)**.
4. After exposing as much of the distal end of the liner as possible, position it against the residual limb and with light compression roll upward onto the limb. Check that no air pockets are present. **(Figure 6)**.
5. Roll the liner all the way up the limb, taking care not to damage it with fingernails. Do not tug or pull.
6. Ensure the seals sit horizontally around their full circumference.

NOTE: Be very careful that the inside of the liner is clean, dry and free from any foreign objects that can cause skin irritation.

NOTE: With use the silicone will soften, which will facilitate application.

DONNING THE PROSTHESIS

The Iceross Seal-In X5 liner is to be used with Icelock Expulsion Valves 551 or 552 (*See ICEROSS® DISTAL CUP*). The valves are specifically designed to maintain suspension when using Seal-In liners. A knee sleeve is not required.

1. Lightly spray the Seals around their full circumference, and inside the hard socket for smoother entry. **(Figure 7)**.
2. Align the prosthesis as instructed by the prosthetist and push into the hard socket expelling air. **(Figure 8)**.

NOTE: If the user is having difficulty donning the prosthesis, more lubricant may be needed or the valve is blocked.

NOTE: If the user feels the skin of the residual limb is being stretched to one side after donning the prosthesis, the distribution of the lubricant is inadequate around the full circumference of the Seals.

VOLUME SOCKS

If volume reduction is experienced in the residual limb, volume socks may be added to restore socket fit.

Volume sock application:

- Pull the volume sock over the Seals and ensure it is placed level between the two most proximal Seals.
- The volume sock can extend over the top two Seals as long as suspension is maintained.
- Don the prosthesis as normal.

NOTE: Placing volume socks over the three distal Seals will result in loss of suspension.

ICEROSS® CLEAN & SIMPLE LUBRICANT SPRAY

Iceross Clean & Simple lubricant spray has to be used to facilitate donning of the liner and the prosthesis.

The lubricant is designed to evaporate quickly after donning to maintain maximum suspension. A spray pen accompanies the Seal-In Liner, small enough to fit in a pocket and/or a handbag. It can be refilled with the Iceross Clean & Simple lubricant spray.

WARNING: Do not apply to the skin. Avoid contact with eyes.

INGREDIENTS: Propanol and water.

ICELOCK® EXPULSION VALVES

The Össur Icelock Expulsion Valves are designed and tested to continually expel air when loading the prosthesis, thus maintaining hypobaric suspension.

Product Number	Description	Benefits
L-551000	Expulsion Valve 551	Small, cosmetic & easy installation
L-552000	Expulsion Valve 552	Removable valve and large smooth opening

ICEROSS® DISTAL CUP

Iceross Distal Cups (I-3000xx) provide enhanced comfort for conical limbs with poor soft tissue coverage. They can also be used as a fitting tool in conjunction with the Iceross Original liners to ensure total contact between the silicone and residual limb. The DERMOGEL silicone in the Iceross Distal Cup conforms to the limb, thus minimizing pressure over bony prominence during gait.

To correctly fit a Distal Cup:

1. Measure the circumference of the residual limb at 4 cm from the distal end. **(Figure 1)**
2. Choose the Distal Cup two or three sizes smaller than the measurement
3. Roll the Distal Cup directly over the limb. **(Figure 9)**
4. When using an Iceross liner in combination with a Distal Cup, the liner measurement (4cm from the distal end) should be made with the Distal Cup in place (i.e. OVER the Distal Cup). **(Figure 10)**
5. Choose the correct Iceross liner size based on the measurement obtained with the Distal Cup in place (See *LINER SIZING*).

ICEROSS® PADS (Figure 11)

Iceross Pads (I-6103xx) can be used to fill in any problem areas (i. e. voids or scars) to ensure better fit and to minimize trapped air. Iceross Pads may also be used to distribute body pressure and to protect the skin. In some cases, custom-formed pads may be required for irregular shapes or

problem areas. See Össur Prosthetic Product Catalogue for more information.

LINER CARE (Figure 12)

Proper product care is important. The liner needs to be washed inside and out every day, following use. Remove the liner, turn the liner inside out and wash it with pH balanced, 100% fragrance- and dye-free soap. The liner can also be machine washed (40 °C/Hot) with a mild detergent. Fabric softeners or bleaches or other products or cleaning solutions may damage the liner and should **NOT** be used. After cleaning, the patient should rinse the liner thoroughly with water and wipe both sides dry with a lint-free cloth.

An Iceross liner can be used directly after washing. It does not need to dry overnight. Always return the liner to its normal position with the distal attachment facing out as soon as possible after cleaning. The Seals must be clean and free from dirt prior to use.

Iceross liners should always be checked for damage or wear. Any damage may weaken the effectiveness of the liner and should be reported to the user's prosthetist immediately. Misuse may result in potential loss of suspension.

Care must be taken by both the prosthetist and the patient not to expose a liner to glass or carbon fibres or other foreign particles. Such substances can become embedded in the silicone causing aggravation of the skin. Washing alone may not be sufficient to eliminate the problem and the patient should be instructed to return the liner to the prosthetist if the liner is inadvertently exposed to any foreign substances or chemicals. The liner silicone contains Aloe Vera extract. This acts as a natural moisturizer and is known for its active healing and soothing effects on the skin. Initially, the inner surface of the liner should be moist to the touch. For these reasons, the liner should not be left inside out for extended period or exposed to excessive heat or sunlight.

SKIN CARE (Figure 13)

Daily cleaning of the residual limb is also essential. We recommend use of a mild liquid, pH balanced, 100% fragrance and dye-free soap. If dry skin is noted, apply a pH balanced, 100% fragrance- and dye-free lotion to nourish and soften the skin. Do **NOT** apply lotion to the residual limb immediately prior to donning the liner.

NOTE: Many common household or bath products, including soaps, deodorants, perfumes, aerosol or alcohol sprays or abrasive cleaners may cause or contribute to skin irritation and the patient should be so advised.

LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

PRODUCT WARRANTY

Össur offers a written warranty on the Iceross Seal-In X5 liner for the

indicated period from the invoice date. Said warranty shall be in lieu of all others, whether express or implied by law.

Iceross Seal-In® X5 – 6 months.

For further information, please contact Össur or your local Össur distributor. You can also visit the Össur web page at www.ossur.com.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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