









Instructions for Use

---

ICEROSS SEAL-IN® LINER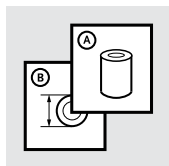

|    |                             |    |
|----|-----------------------------|----|
|    |                             | 3  |
| EN | Instructions for Use        | 4  |
| DE | Gebrauchsanweisung          | 8  |
| FR | Notice d'utilisation        | 12 |
| ES | Instrucciones para el uso   | 16 |
| IT | Istruzioni per l'uso        | 20 |
| NO | Bruksanvisning              | 24 |
| DA | Brugsanvisning              | 28 |
| SV | Bruksanvisning              | 31 |
| EL | Οδηγίες χρήσης              | 35 |
| FI | Käyttöohjeet                | 39 |
| NL | Gebruiksaanwijzing          | 43 |
| PT | Instruções de Utilização    | 47 |
| PL | Instrukcja użytkowania      | 51 |
| CS | Návod k použití             | 55 |
| TR | Kullanım Talimatları        | 59 |
| RU | Инструкция по использованию | 63 |
| JA | 取扱説明書                       | 67 |
| ZH | 中文说明书                       | 71 |
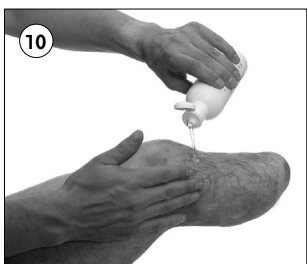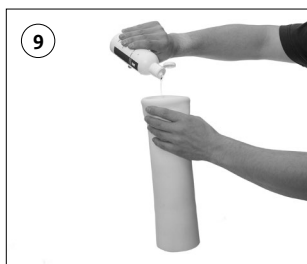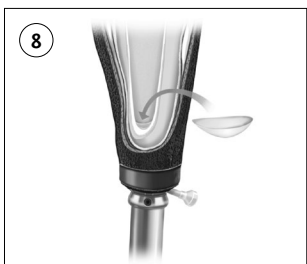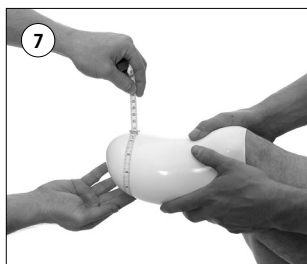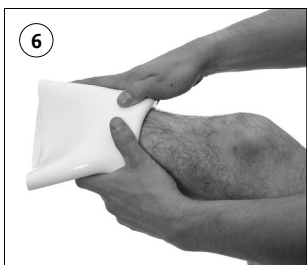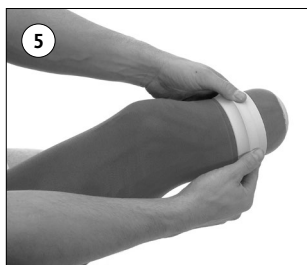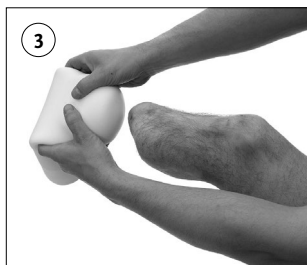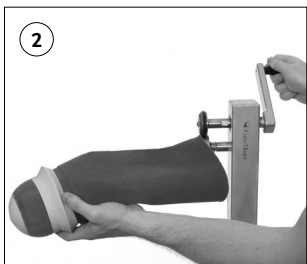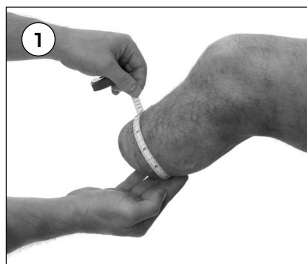
## INTRODUCTION

Thank you for choosing a high quality silicone liner from Össur. We can assure you that you have made the right choice. Please carefully review and follow these instructions before fitting a patient with the liner or accessory products. Remember that informing the patient concerning proper handling and care of the liner, as set forth in these instructions, is important to achieve patient comfort and maximum product life.

## PRODUCT SELECTION AND FITTING (Figure 1)

### *Liner Sizing*

To determine the correct Iceross size:

1. Measure the circumference of the residual limb at 4cm from the distal end.
2. Choose the Iceross size, which corresponds to the measurement or which, is immediately below the indicated measurement.
3. Never choose a liner size, which exceeds the measurement, or round up.

**Example:** If the residual limb measures 24.5cm at the indicated location, choose Iceross size 23.5 NOT size 25.

It is essential that the Hypobaric Sealing Membrane (HSM™) sits correctly against the liner around its full circumference. To ensure the HSM is correctly positioned, run your fingers around the seal and assess the tension.

To ensure optimal suspension with the Iceross Seal-In liner there must be a slight tension around the proximal edge of the HSM seal. If the seal is at all loose than a crease may form when donning the socket, this may be prevented by selecting the next liner size below. If the liner is too tight, this may cause pistoning and/or numbness and blistering. A Seal-In Liner that is too tight will also produce excessive ring tension to the residual limb from the HSM. If the liner is too loose, increased perspiration and/or movement of the limb inside the liner may occur. Perspiration and movement can lead to blisters and rashes. If any of the above symptoms or any other indication of improper fit are noted, the user should be instructed to contact his/her physician or prosthetist immediately.

**Note:** The matrix present within the liner extends 10cm from the distal end of the liner. Ensure that this matrix does not extend over the fibular head.

### *Trimming an Iceross Liner* (Figure 2)

Iceross liners may be trimmed for user preference and to increase range of motion. The Iceross CurveMaster is specifically designed for this function and to produce a smooth, rounded edge, which will minimize the risk of skin irritation and/or tears of the liner. Do not cut the liner below the level of the matrix or below the proximal trim-lines of the socket. Excessive trimming may reduce the natural suction between the liner and limb and/or compromise suspension.

### ***Donning an Iceross Liner***

- When donning the liner, turn the liner inside out and grip it as shown in the illustration (Figure 3).

**Note:** Be very careful that the inside of the liner is clean, dry and free from any foreign objects that can cause skin irritation.

- After exposing as much of the distal end of the liner as possible, position it against the residual limb and with light compression roll upward onto the limb. Check that no air pockets are present (Figure 4).
- Roll the liner all the way up the limb, taking care not to damage it with fingernails. Do not tug or pull (Figure 5).
- Ensure the Silicone Membrane is not inverted and sits correctly around its full circumference (Figure 6).

**Note:** Inverting the Iceross Seal-In Liner and returning it to its original position will soften the silicone and assist with donning of the liner when new.

### ***Iceross Distal Cup***

Iceross Distal Cups provide enhanced comfort for conical limbs with poor soft tissue coverage. They can also be used as a fitting tool in conjunction with the Iceross Original liners to ensure total contact between the silicone and residual limb. The DERMOGEL silicone in the Iceross Distal Cup conforms to the limb, thus minimizing pressure over bony prominence during gait.

*To correctly fit a Distal Cup:*

1. Measure the circumference of the residual limb at 4cm from the distal end (Figur 1).
2. Choose the Distal Cup two or three sizes smaller than the measurement.
3. Roll the Distal Cup directly over the limb (Figure 6).
4. When using an Iceross liner in combination with a Distal Cup, the liner measurement (4cm from the distal end) should be made with the Distal Cup in place (i.e. OVER the Distal Cup) (Figure 7).
5. Choose the correct Iceross liner size based on the measurement obtained with the Distal Cup in place

### ***Iceross Pads*** (Figure 8)

Iceross Pads can be used to fill in any problem areas (i.e. voids or scars) to ensure better fit and to minimize trapped air. Iceross Pads may also be used to distribute body pressure and to protect the skin. In some cases, custom-formed pads may be required for irregular shapes or problem areas.

### ***Liner Care*** (Figure 9)

Proper product care is important. The liner needs to be washed inside and out every day, following use. Your patient should be instructed to remove the liner, turn the liner inside out and wash it with pH balanced, 100% fragrance- and dye-free soap. The liner can also be machine washed (40°C/Hot) with a mild detergent. Fabric softeners or bleaches or other products or cleaning solutions may damage the liner and should NOT be used. After cleaning, the patient should rinse the liner thoroughly with water and wipe both sides dry with a lint-free cloth.

An Iceross liner can be used directly after washing. It does not need to dry over night. Always return the liner to its normal position with the distal attachment facing out as soon as possible after cleaning. Ensure the Silicone Membrane is positioned correctly on the liner following washing. The Silicone Membrane must be clean and free from dirt prior to use.

**Note:** Misuse may result in potential loss of suspension. Iceross liners should always be checked for damage or wear. Any damage may weaken the effectiveness of the liner and should be reported to the user's prosthetist immediately. Care must be taken by both the prosthetist and the patient not to expose a liner to glass or carbon fibers or other foreign particles. Such substances can become embedded in the silicone causing aggravation of the skin. Washing alone may not be sufficient to eliminate the problem and the patient should be instructed to return the liner to the prosthetist if the liner is inadvertently exposed to any foreign substances or chemicals.

DermoSil and DermoGel silicone contain Aloe Vera extract. This acts as a natural moisturizer and is known for its active healing and soothing effects on skin. Initially, the inner surface of the liner should be moist to the touch. For these reasons, the liner should not be left inside out for extended period or exposed to excessive heat or sunlight.

### ***Skin Care (Figure 10)***

Daily cleaning of the residual limb is also essential. We recommend use of a mild liquid, pH balanced, 100% fragrance and dye-free soap.

If dry skin is noted, apply a pH balanced, 100% fragrance- and dye-free lotion to nourish and soften the skin.

Do **NOT** apply lotion to the residual limb immediately prior to donning the liner.

**Note:** Many common household or bath products, including soaps, deodorants, perfumes, aerosol or alcohol sprays or abrasive cleaners may cause or contribute to skin irritation and the patient should be so advised.

### ***Product Warranty***

Össur offers a written warranty on the Iceross Seal-In Liner for the indicated period from the invoice date. Said warranty shall be in lieu of all others, whether express or implied by law.

|                       |          |
|-----------------------|----------|
| Iceross Seal-In Liner | 6 months |
|-----------------------|----------|

### **ICEROSS SEAL-IN SOCK**

If volume reduction is experienced in the residual limb, Seal-in socks may be added to restore the socket fit.

#### ***Seal-in sock application:***

- Invert the HSM seal completely in the distal direction.
- Pull the Seal-in sock over the inverted HSM seal until the distal end of the sock is level with the proximal border of the seal.
- Reflect the HSM seal over the end of the sock.
- Don the prosthesis as normal.

**Note:** Loss of suspension may be encountered if a sock is placed over the

HSM seal. The Seal-in Liner will function with up to 6-ply thickness (2 x 3ply).

To be washed at 50°C with light coloured fabric.  
Do not tumble dry.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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