





Instructions for Use

ICEROSS[®] SLEEVE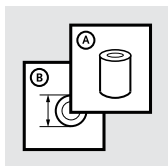

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	5
FR	Notice d'utilisation	6
ES	Instrucciones para el uso	7
IT	Istruzioni per l'uso	8
DA	Brugsanvisning	9
SV	Bruksanvisning	10
EL	Οδηγίες Χρήσης	11
NL	Gebruiksaanwijzing	12
PT	Instruções de Utilização	13
JA	取扱説明書	14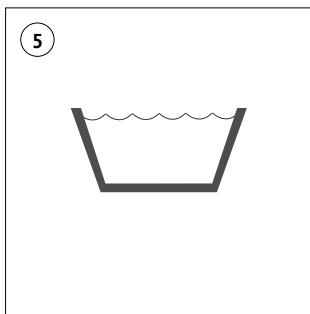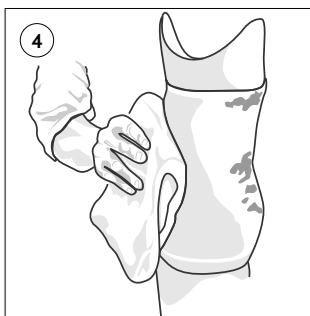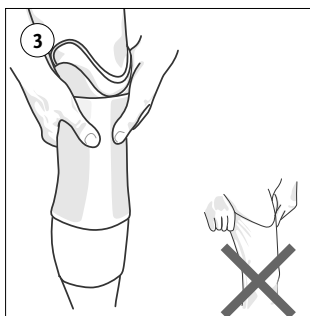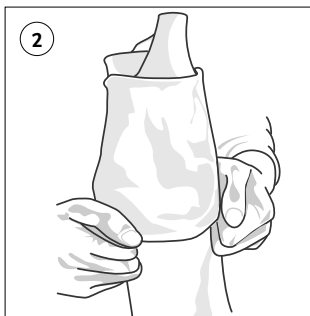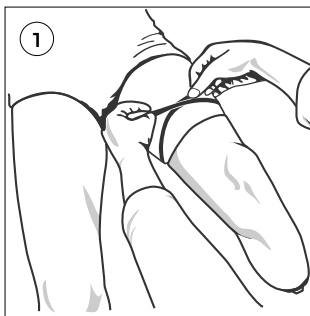

ENGLISH

1. Measure circumference 15cm (6") above mid-patella and choose the correct size. See sizing chart below (**Figure 1**).

Size	Measurement (cm)	Measurement (inches)
35	35 - 41 cm	13 ³ / ₄ - 16"
41	41 - 49 cm	16 - 19 ¹ / ₄ "
49	49 - 55 cm	19 ¹ / ₄ - 21 ³ / ₄ "
55	55 - 62 cm	21 ³ / ₄ - 24 ¹ / ₂ "
62	62 - 76 cm	24 ¹ / ₂ - 30"

2. Turn the Iceross sleeve inside out, half way, with silicone of distal end half-exposed. Stretch the sleeve over the brim of the socket and roll distal end completely down (**Figure 2**).
3. Roll down upper part of the sleeve until socket brim is completely exposed. Step into the socket and make sure the residual limb is positioned correctly in the socket. Roll the sleeve up with both hands. Do not pull on the edges as this may place strain on the skin (**Figure 3**).
4. Wash the Iceross sleeve daily, especially on the inside. To do so, remove the sleeve, turn the liner inside out and wash it with pH balanced, 100% fragrance- and dye-free soap. Be sure to rinse with plenty of water, and wipe dry with a towel.
The sleeve can also be rolled down onto the socket and washed (**Figure 4**).
5. The Iceross sleeve can also be machine washed (40-60°C/Warm). Turn the sleeve inside out during washing. Rinse thoroughly with clean water to remove all soap residue (**Figure 5**).
6. Sealant loops on the inside of the liner should be located on different places on the thigh every other day to prevent skin irritation at the location of the sealant loops.

All Iceross sleeves are 35cm long (14").

WARRANTY: Iceross Sleeve comes with a 90-day warranty against defects in material and workmanship.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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