
Instructions for Use





---

BALANCE™ FOOT J

JBPExyz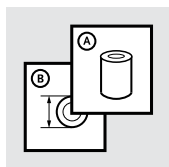


		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	9
ES	Instrucciones para el uso	11
IT	Istruzioni per l'uso	13
NO	Bruksanvisning	15
DA	Brugsanvisning	17
SV	Bruksanvisning	19
EL	Οδηγίες χρήσης	21
FI	Käyttöohjeet	23
NL	Gebruiksaanwijzing	25
PT	Instruções de Utilização	27
JA	取扱説明書	29
ZH	中文说明书	31
KO	사용 설명서	33

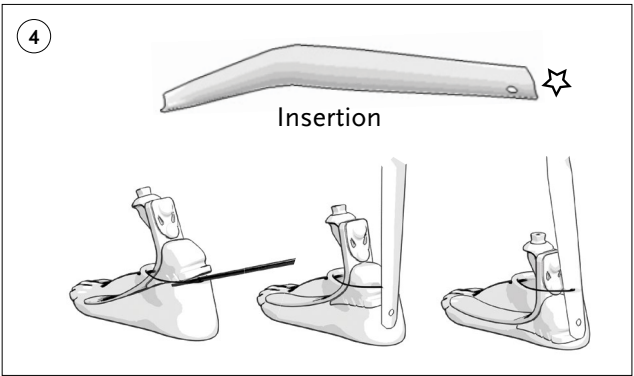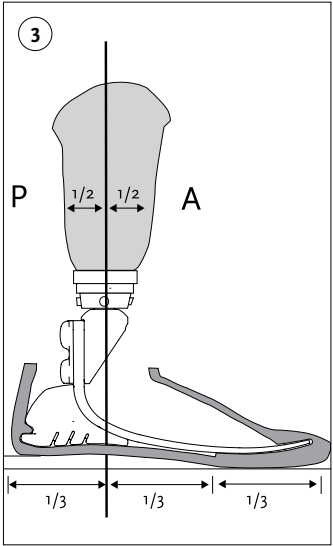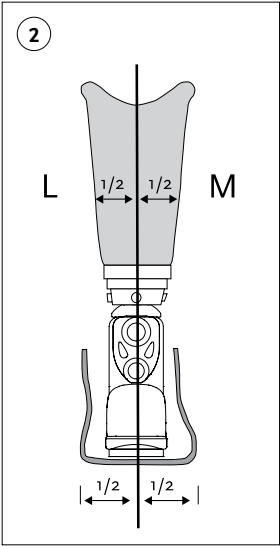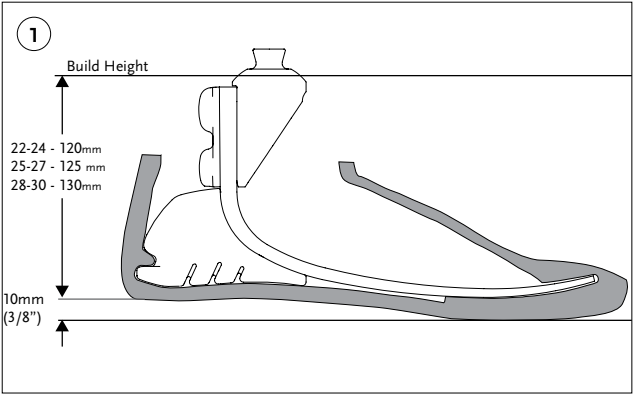5

Removal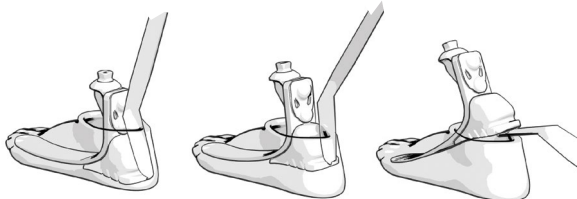

# ENGLISH

---

## TECHNICAL SPECIFICATIONS (Figure 1)

- Male pyramid connection
- Clearance: 120mm (22-24)  
125mm (25-27)  
130mm (28-30)
- Net weight: Size 27, cat2 w/ pyramid: 322g
- Foot cover size 27: 254g
- MAX 136kg (300lbs).
- Available in sizes 22-30cm.

## INDICATIONS FOR USE

- Balance Foot J is rated for low impact levels (community ambulators).
- Not suitable for sport and high-impact activities.

## ALIGNMENT: Figure 2

### BENCH ALIGNMENT (Figure 2 & 3)

1. Fit the foot cover (Figure 3 & 4).
2. Adjust to appropriate heel height (using the shoe).
3. Introduce appropriate socket angles.
4. The load line should fall through the 1/3rd mark on the brim of the foot cover. The mark represents the junction of the posterior and the middle third portion of the foot.

### Dynamic Alignment

The foam heel provides cushioning at heel strike. The toe provides forward progression at toe off.

### The heel to toe action can be influenced by:

- A-P positioning of foot
- Dorsi-Plantarflexion

## CATEGORY SELECTION CHART

Please refer to the selection charts below to determine the appropriate stiffness required according to Össur recommendations.

WEIGHT KG	45-61	62-80	81-97	98-114	115-136
WEIGHT LBS	99-135	136-176	177-214	215-251	252-300
22	1	2	N/A	N/A	N/A
23	1	2	3	4	N/A
24	1	2	3	4	N/A
25	1	2	3	4	5
26	1	2	3	4	5
27	1	2	3	4	5
28	N/A	2	3	4	5
29	N/A	N/A	3	4	5
30	N/A	N/A	N/A	4	5

## SHOEHORN

When removing and installing the cover use shoehorn to avoid damage to foot or cover (Figure 4 & 5).

## WEATHERPROOF


A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

## LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

## COMPLIANCE

This component has been tested according to ISO 10328 standard to two million load cycles. Depending on the amputee's activity this corresponds to a duration of use of two to three years. We recommend carrying out regular yearly safety checks

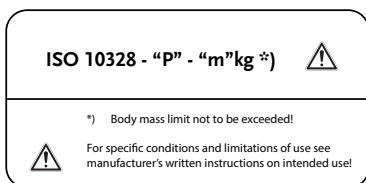In the standard mentioned, test levels (P) are assigned to a certain maximal body masses (m in kg). In some cases, which are marked with, no test level is assigned to the product related maximal body mass.

In these cases, the test loads have been adapted adequately on the basis of the specified load level.

ISO10328					
Category	Weight (Kg)	Label text			
0	44	ISO 10328 -	P3	44	kg
1	66	ISO 10328 -	P3	61	kg
2	80	ISO 10328 -	P4	80	kg
3	97	ISO 10328 -	P5	97	kg
4	114	ISO 10328 -	P5	114	kg
5	136	ISO 10328 -	P6	136	kg

- This manual is intended for use by a certified prosthetist.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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