



Instructions for Use

PRO-FLEX[®]

Product number: PFP0xyz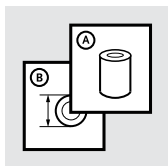

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	14
ES	Instrucciones para el uso	19
IT	Istruzioni per l'uso	24
NO	Bruksanvisning	29
DA	Brugsanvisning	34
SV	Bruksanvisning	39
EL	Οδηγίες χρήσης	44
FI	Käyttöohjeet	49
NL	Gebruiksaanwijzing	54
PT	Instruções de Utilização	59
PL	Instrukcja użytkowania	64
JA	取扱説明書	69
KO	사용 설명서	74

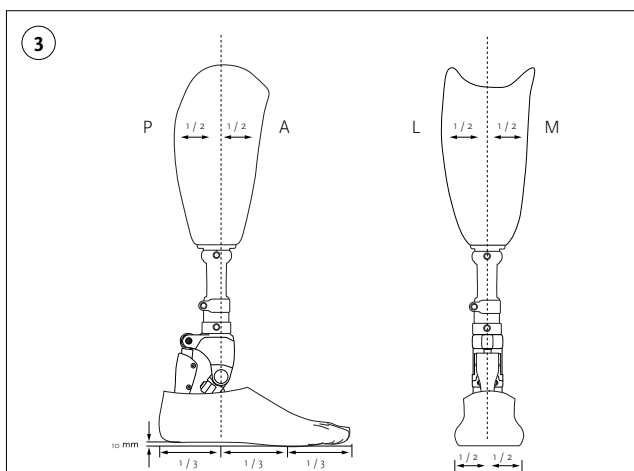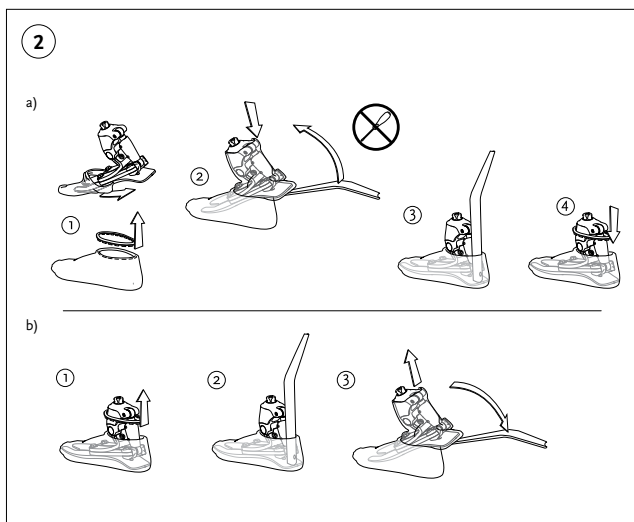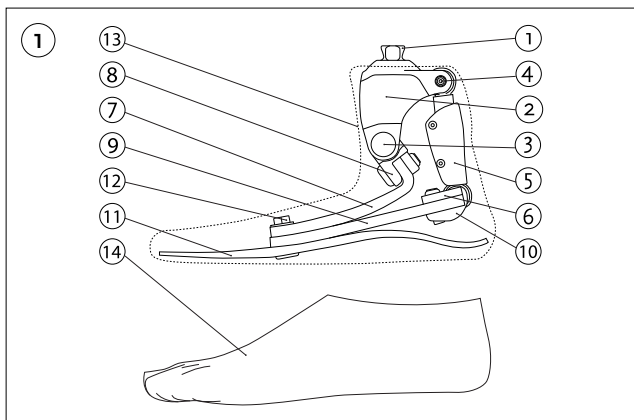ENGLISH

This manual is intended for use by a certified Prosthetist.

SAFETY PRECAUTIONS

Caution: Statement that alerts the user to the possibility of a problem with the device associated with its use or misuse.

Precaution: Should be taken to avoid the hazard

Note: Possibility of a technical damage or malfunction of the device.

Caution: Make sure that the user knows if he/she feels or notices any change of function or alignment of the device, he should contact his healthcare practitioner. This noticeable function loss may be the result of reduced stiffness or loss of support.

Note: Adjusting the rear link is not allowed.

Note: Make sure that the user knows that he needs to replace socks that have holes in it because otherwise it may damage the foot cover.

Note: The foot is designed to be used with spectra socks and foot cover to prevent noise and for proper shoe fit.

CAUTION! Avoid placing hands or fingers near moving joints.

PRODUCT DESCRIPTION

PRO-FLEX is a prosthetic foot with anatomically located ankle pivot joint. Two supporting joints connect flexible carbon blades and act together with the main joint to provide progressive energy storing during the whole stance phase and a powerful push-off in the end of stance.

The device consists of the following components (**Figure 1**):

1. Male Pyramid
2. Chassis
3. Main pivot
4. Upper supporting pivot
5. Supporting link
6. Lower supporting pivot
7. Top blade
8. Main pivot carbon clamp
9. Middle blade
10. Lower supporting pivot carbon clamp
11. Bottom blade
12. Bottom blade bolts
13. Sock
14. Footcover

FOOT COVERS (Figure 2)

The foot is designed to be used with the FST Foot Covers and spectra socks. Please view the instructions for use for the foot covers before fitting them and ensure that the user knows how to use them.

Use a shoehorn to avoid damaging to the foot or the foot cover.

- a. Installing – Make sure that the foot module is placed completely into the foot cover.
- b. Removing the foot from the foot cover.

Note: Use spectra socks as shown in figure 1.

INDICATIONS FOR USE

The foot is a non-invasive, single patient, reusable prosthetic device. Recommended to be used by low - moderate active user in everyday use.

MEDICAL INDICATION

Lower extremity amputations, for example due to:

- Trauma
- Vascular Disease
- Cancer
- Congenital Defects

CONTRAINDICATIONS FOR USE

The foot should not be used in any sport activities.

TECHNICAL SPECIFICATIONS

Following specs are for a size 27 cat 5 foot:

- Product weight with footcover : 922 g
- Foot cover weight: 165 g
- Build height: 156 mm
- Maximum user weight: 125 kg / 275 lbs.
- Foot size 22–30

CATEGORY SELECTION CHART

Please refer to the selection charts below to determine the appropriate stiffness required according to Össur recommendations.

Weight kg	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-125
Weight lbs	99-115	116-130	131-150	151-170	171-194	195-221	222-256	257-275
Low activity - Category								
Size 22	1	1	2	3	4	N/A	N/A	N/A
Sizes 23-24	1	1	2	3	4	5	N/A	N/A
Sizes 25-26	2	2	2	3	4	5	6	7
Sizes 27-28	N/A	N/A	N/A	3	4	5	6	7
Sizes 29-30	N/A	N/A	N/A	N/A	4	5	6	7
Moderate activity - Category								
Size 22	1	2	3	4	N/A	N/A	N/A	N/A
Sizes 23-24	1	2	3	4	5	N/A	N/A	N/A
Sizes 25-26	2	2	3	4	5	6	7	N/A
Sizes 27-28	N/A	N/A	3	4	5	6	7	N/A
Sizes 29-30	N/A	N/A	N/A	4	5	6	7	N/A

ALIGNMENT FOR THE DEVICE

Note: Remove the yellow sticker on the top of the pyramid before you align the device.

BENCH ALIGNMENT (Figure 3)

- Fit foot with spectra sock and selected foot cover.
- Adjust to appropriate heel height (using the shoe).

- Introduce appropriate socket angles: flexion/extension and abduction/adduction.
- Divide the foot cover into 3 equal portions.
- The load line should fall in the middle section on the 1/3 line, or slightly in anterior.

DYNAMIC ALIGNMENT

The foot provides progressive energy storing during the whole stance phase and a powerful push-off in the end of stance. Because of this it is expected that under load the load line will move anteriorly to the 1/3 line. The heel to toe action can be influenced by:

- Shoe performance
- A-P positioning of foot
- Dorsi-Plantarflexion

Note: For Transfemoral users please follow the recommendations for the alignment of the corresponding Össur knee joint.

HEEL STIFFNESS

Heel too soft

Symptoms:

- Foot will come to flat position too early (User feels he/she is sinking into a hole)
- Extra energy is required to climb up over the toe.
- Toe will feel too stiff
- Knee may hyperextend

Actions:

- Shift socket anterior (or foot posterior)

Heel too hard

Symptoms:

- Rapid heel to toe movement.
- At initial contact the amputee has poor control of his prosthesis
- Minimal energy return feeling.
- Knee may become unstable.

Actions:

- Shift socket posterior (or foot anterior).
- Reconsider shoe selection

Foot too soft/stiff

Symptoms:

- Foot too soft/stiff

Actions:

- Reconsider category selection

Foot size not matching sound side

Symptoms:

- Foot moving inside of shoe

Actions:

- Change foot to a correct size

MAINTAINANCE & CLEANING

Note: The device should always be used with spectra socks and foot covers to minimize dust and dirt from entering between carbon blades. It is important to fix the spectra sock around pyramid location in order to prevent noise issues coming from sand entering between the plates.

The device is weather-proof but not resistant to corrosion. Therefore, the device should not come into contact with salt water or chlorinated water. The device is not designed to be used under extreme conditions like diving or jumping into water.

If for some reason the foot components get wet parts should be dried with a lint free cloth.

Should the device come into contact with salt water, chlorinated water, dust or dirt it must be rinsed with fresh water immediately.

USE ENVIRONMENT

- Environmental operating temperature: -15°C (+5°F) to 50°C (122°F).
- Operating humidity: 10%-95% Relative humidity.
- Shipping and storage temperature: -20°C (-4°F) to 70°C (158°F).

WEATHERPROOF

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.


LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

COMPLIANCE

This component has been tested according to ISO 10328 standard to two million load cycles. Depending on the amputee's activity this corresponds to a duration of use of two to three years. We recommend carrying out regular yearly safety checks. In the standard mentioned, test levels (P) are assigned

to a certain maximal body masses (m in kg). In some cases, which are marked with, no test level is assigned to the product related maximal body mass. In these cases, the test loads have been adapted adequately on the basis of the specified load level.

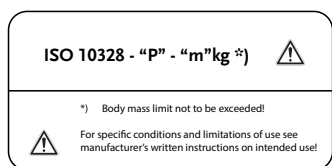ISO 10328 - label		
Category	Weight (Kg)	Label text
1	52	ISO 10328-P3-52-kg
2	59	ISO 10328-P3-59-kg
3	68	ISO 10328-P3-68-kg
4	77	ISO 10328-P4-77-kg
5	88	ISO 10328-P4-88-kg
6	100	ISO 10328-P5-100-kg
7	116	ISO 10328-P6-116-kg

WARRANTY

Össur provides a warranty of 36 months for this foot module and 6 months for the foot cover.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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