



Instructions for Use



---

PRO-FLEX LP TORSION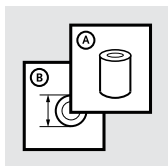

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	10
FR	Notice d'utilisation	15
ES	Instrucciones para el uso	20
IT	Istruzioni per l'uso	25
NO	Bruksanvisning	30
DA	Brugsanvisning	35
SV	Bruksanvisning	40
EL	Οδηγίες Χρήσης	45
FI	Käyttöohjeet	50
NL	Gebruiksaanwijzing	55
PT	Instruções de Utilização	60
PL	Instrukcja użytkowania	65
JA	取扱説明書	70
KO	사용 설명서	74
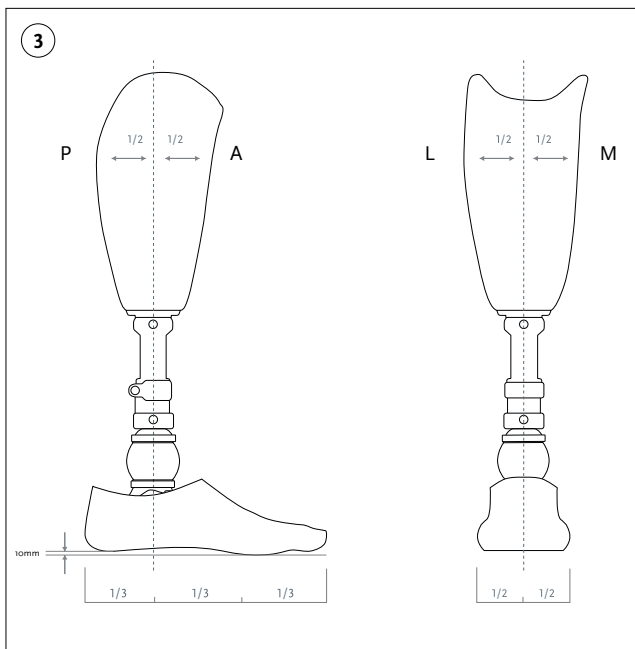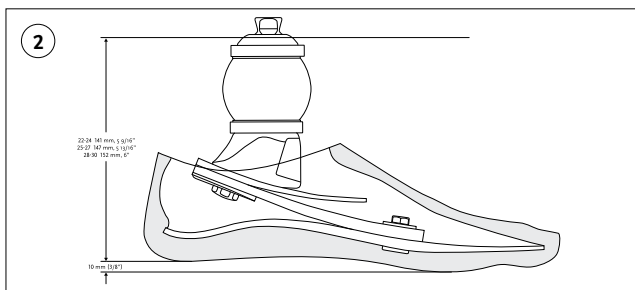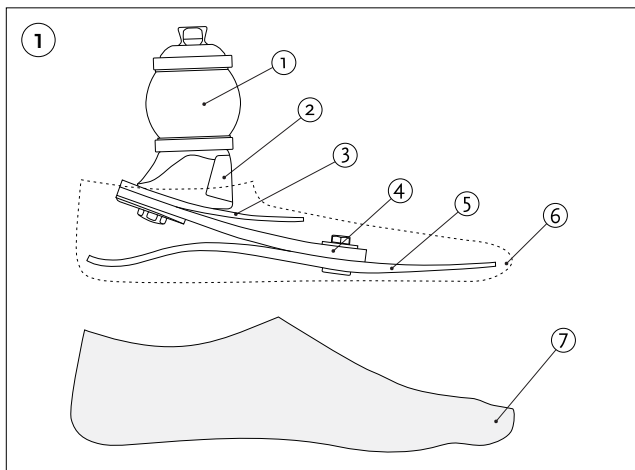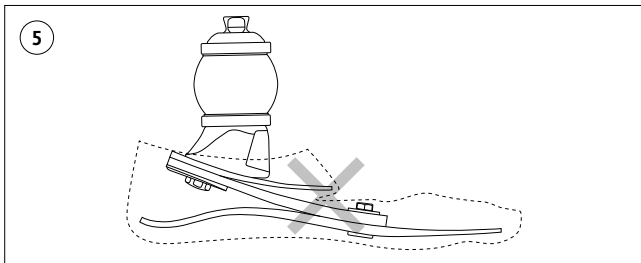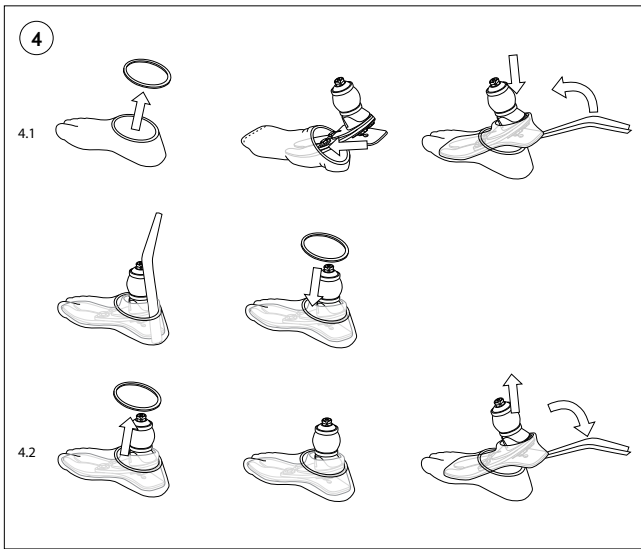
# ENGLISH

---

**Pro-Flex LP Torsion** is referred to as **the device** in the following document. This document provides important information on the indications for use, and handling of the device. It is intended for use by a certified prosthetist and the user of the device. The device may only be fitted by a qualified practitioner authorized by Össur after completing the corresponding training.

## PRODUCT DESCRIPTION

The device is a mechanical foot with an integrated torsion and shock absorbing adapter, and male pyramid.

The device consists of the following components: **(Figure 1)**

1. Torsion shock assembly
2. Pyramid cover
3. Top blade
4. Middle blade
5. Bottom blade
6. Spectra Sock
7. Foot cover

## INDICATIONS FOR USE

The device is a non-invasive, reusable prosthetic device.

The device is designed for single patient use in low to high everyday activities (e.g. walking).

The device is not designed for use in extreme sports.

The device is tested and validated for exoprosthetic fittings of lower extremity amputations.

## CONTRAINDICATIONS

Not applicable

## TECHNICAL SPECIFICATIONS (Figure 2)

The following specifications are for a size 27 category 5 foot with foot cover:

- Product weight with footcover : 960 g / 2.11 lbs
- Foot cover weight: 165 g / 0.36 lbs
- Build height: 147 mm / 5 25/32"
- Maximum user weight: 147 kg / 324 lbs

## CATEGORY SELECTION CHART

Please refer to the selection chart below to determine the appropriate category required according to Össur recommendations.

**NB:** Incorrect category selection may result in poor device function. The weight limit is 147 kg for low active users, 130 kg for moderate active users, and 116 kg for high active users.

Weight kg	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130	131-147
Weight lbs	99-115	116-130	131-150	151-170	171-194	195-220	221-256	257-287	288-324
Low Impact - Category									
Sizes 22-24	1	1	2	3	4	5	N/A	N/A	N/A
Sizes 25-27	1	1	2	3	4	5	6	7	8
Sizes 28-30	3	3	3	3	4	5	6	7	8
Moderate Impact - Category									
Sizes 22-24	1	2	3	4	5	N/A	N/A	N/A	N/A
Sizes 25-27	1	2	3	4	5	6	7	8	N/A
Sizes 28-30	3	3	3	4	5	6	7	8	N/A
High Impact - Category									
Sizes 22-24	2	3	4	5	N/A	N/A	N/A	N/A	N/A
Sizes 25-27	2	3	4	5	6	7	8	N/A	N/A
Sizes 28-30	3	3	4	5	6	7	8	N/A	N/A

## SAFETY

**Warning:** A strong cautionary statement regarding possible dangers related to the intended use or foreseeable misuse of a device.

**Precaution:** A cautionary statement requiring action to be taken in advance of device use, or in the event of malfunction or change in device performance. Intended to ensure safe and effective use of device, and to protect the user against greater risk of harm and/or possible failure of the device, when used as intended.

## WARNINGS

Össur products and components are designed and tested according to applicable official standards, or an in-house defined standard where no official standard applies. Compatibility and compliance with these standards is achieved only when Össur products are used with recommended Össur components.

The device should be used with Össur adapters. The durability of the device cannot be guaranteed when non-Össur adapters are used.

In cases of damage, malfunction, changes in alignment, or functional loss, the user should immediately cease using the device, and return it to their practitioner. Noticeable functional loss may be the result of reduced stiffness or loss of support.

Improper handling (including repetitive high loads) can cause device malfunction.

The device is weatherproof, therefore cannot be submerged in water (please refer to the Weatherproof section).

## PRECAUTIONS

Use of a shoe horn is recommended when donning or doffing the foot cover, to prevent damage to both the device and foot cover.

The user must keep fingers clear of the blades when in use, as a pinch hazard exists between blades.

The protective film on the pyramid must be removed when device fitting is complete.

**NB:** The device is designed to be used with a Spectra sock, and recommended foot cover. The device should not be used without these accessories.

## **WEATHERPROOF**

A weatherproof device allows use in a wet and/or humid environment, but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

For a weatherproof system, weatherproof components must be selected for use with the device.

## **BENCH ALIGNMENT (Figure 3)**

- Fit foot with selected cover and shoe.
- Adjust to appropriate heel height (using the shoe).
- Ensure appropriate angles (flexion/extension and abduction/adduction).
- Divide the foot cover into 3 equal portions.
- The load line should fall at the junction of the posterior and the middle third portions as shown in figure 3.
- The 1/3 position is marked on the inside wall of the foot cover.

## **DYNAMIC ALIGNMENT**

The heel stores energy after initial contact, slowly releasing it at mid-stance. The forward momentum generated by this action results in the toe being loaded for optimum energy release at terminal stance.

The heel-to-toe action can be influenced by:

- Anterior/posterior positioning of the foot
- Dorsi-plantarflexion
- Heel stiffness

## **HEEL STIFFNESS**

### **Heel too soft**

#### *Symptoms*

- Foot comes to a flat position too early (user feels they are sinking into a hole).
- Extra energy required to climb up over the toe.
- Toe feels too stiff.
- Knee may hyperextend.

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### **Heel too hard**

#### *Symptoms*

- Rapid heel-to-toe movement.
- At initial contact the user has poor control of prosthesis.
- User feels minimal energy return.
- Knee may become unstable.

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## FOOT COVERS AND SPECTRA SOCKS (Figure 4)

- The device is designed to be used with a recommended foot cover and Spectra sock.
- Use a shoehorn when fitting and removing foot covers to avoid damage (Figure 4.1 and 4.2).
- Ensure the device is placed completely into the foot cover when fitting.
- The middle section of the Spectra sock must cover the top blade of the device to ensure it doesn't interfere with the movement of the blades (Figure 5).

## MAINTAINANCE & CLEANING

- The device is weatherproof, but not resistant to corrosion. The device should not come into contact with salt water or chlorinated water. In cases where it does, the device must be rinsed with fresh water immediately, and dried with a lint-free cloth.
- The device is not designed to be used in extreme conditions such as diving or jumping into water.
- If device components get wet, parts should be dried with a lint-free cloth.

## Debris between blades

- Noise may occur if sand or debris are present between blades.
- The prosthetist should remove the Spectra sock and foot cover from the device, and use compressed air to remove sand or debris.

**Note:** The device should always be used with a foot cover and Spectra sock to minimize dust and dirt from entering between carbon blades.

## ENVIRONMENTAL CONDITIONS


- Environmental Operating Temperature: -15°C (-5°F) to 50°C (122°F).
- Operating Humidity: 10% - 95% Relative Humidity.
- Shipping and Storage Temperature: -20°C (-4°F) to 70°C (158°F).

## LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

## COMPLIANCE

This device has been tested according to ISO 10328 standard to two million load cycles. Depending on the user's activity this corresponds to a duration of use of two to three years. We recommend carrying out yearly safety checks.

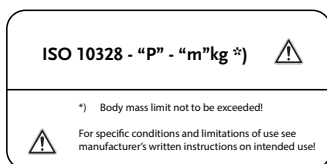ISO 10328 - label		
Category	Weight (Kg)	Label text
1	52	ISO 10328-P3-52-kg
2	59	ISO 10328-P3-59-kg
3	68	ISO 10328-P3-68-kg
4	77	ISO 10328-P4-77-kg
5	88	ISO 10328-P4-88-kg
6	100	ISO 10328-P5-100-kg
7	116	ISO 10328-P6-116-kg
8	130	ISO 10328-P6-130-kg

#### **WARRANTY**

Össur provides a warranty of 36 months for the device, and 6 months for the foot cover.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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