Instructions for Use

PRO-FLEX[®] XC



Product number:

PXC0xyz



PXCUxyz


		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	13
ES	Instrucciones para el uso	17
IT	Istruzioni per l'uso	21
NO	Bruksanvisning	25
DA	Brugsanvisning	29
SV	Bruksanvisning	33
EL	Οδηγίες χρήσης	37
FI	Käyttöohjeet	41
NL	Gebruiksaanwijzing	45
PT	Instruções de Utilização	49
PL	Instrukcja użytkownika	53
JA	取扱説明書	57
KO	사용 설명서	61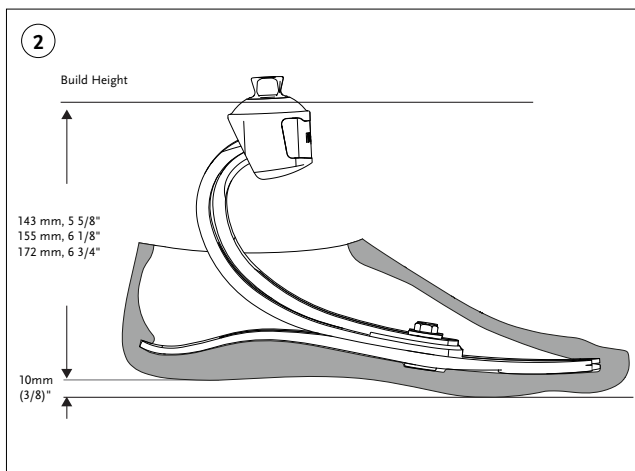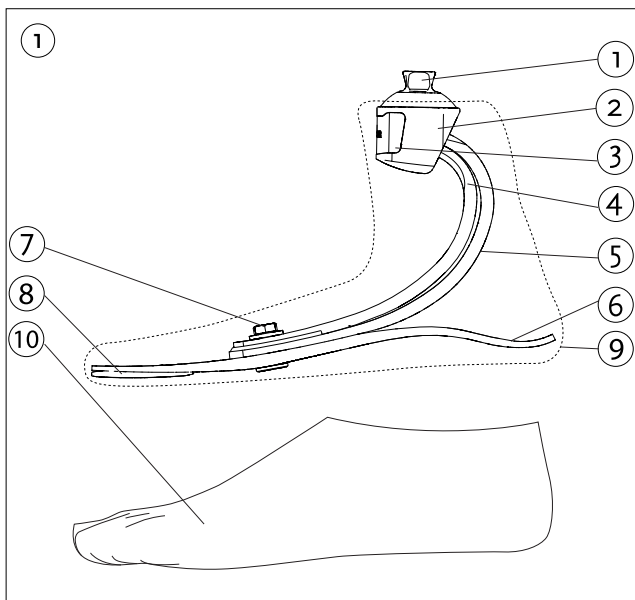

3

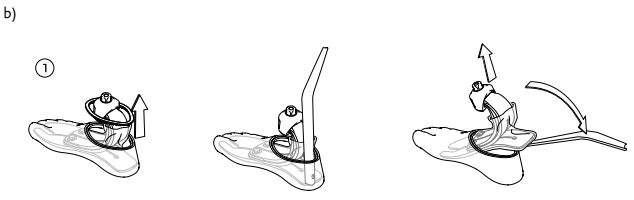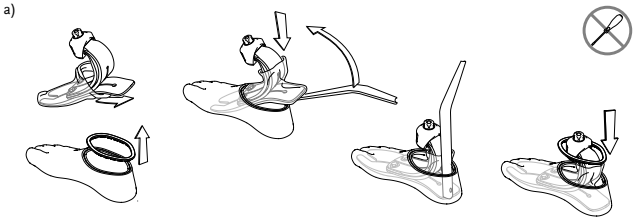4

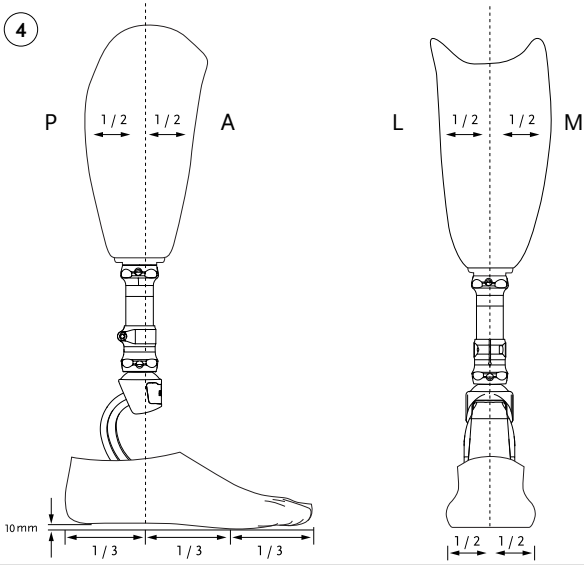ENGLISH

This manual is intended for use by a certified Prosthetist.

PRODUCT DESCRIPTION

The Pro-Flex® XC is a prosthetic foot device with integrated male pyramid. It is composed of a dual blade and a bottom blade. Those parts are secured together using heel bolts. The bottom blade features a sandal toe and a full length composite blade.

The device consists of the following components **(Figure 1)**:

1. Male Pyramid Adapter
2. Pyramid housing
3. Pyramid cover
4. Upper C blade
5. Lower C blade
6. Bottom blade
7. Bottom blade bolts
8. Drop toe
9. Spectra Sock
10. Footcover

FOOT COVERS (Figure 3)

The foot is designed to be used with foot covers and spectra socks. When installing and removing the foot covers use shoehorn to avoid damage to the foot or the foot cover.

Please view the instructions for use for the foot covers before fitting them and ensure that the user knows how to use them.

INDICATIONS FOR USE

The foot is a non-invasive, single patient, reusable prosthetic device. The Pro-Flex XC is tested and validated for exoskeletal prosthetic fittings of lower extremities amputations.

MEDICAL INDICATION

Lower extremity amputations, for example due to:

- Trauma
- Vascular Disease
- Cancer
- Congenital Defects

TECHNICAL SPECIFICATIONS

Following specs are for a size 27 cat 5 foot:

- Product weight with footcover : 670 g / 1,47lbs
- Foot cover weight: 165 g / 0,36 lbs
- Build height: 155 mm / 6 1/8"
- Maximum user weight: 166 kg / 365 lbs

CATEGORY SELECTION CHART

Please refer to the selection charts below to determine the appropriate stiffness required according to Össur recommendations.

Weight kg	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130	131-147	148-166
Weight lbs	99-115	116-130	131-150	151-170	171-194	195-221	222-256	257-275	288-324	325-365
Low Impact Level	1	1	2	3	4	5	6	7	8	9
Moderate Impact Level	1	2	3	4	5	6	7	8	9	N/A
High Impact Level	2	3	4	5	6	7	8	9	N/A	N/A

- Foot size 22–30

SAFETY PRECAUTIONS

Caution: Statement that alerts the user to the possibility of a problem with the device associated with its use or misuse.

Precaution: Should be taken to avoid the hazard

Note: Possibility of a technical damage or malfunction of the device.

Caution: Make sure that the user knows if he/she feels or notices any change of function or alignment of the device, he should contact his healthcare practitioner. This noticeable function loss may be the result of reduced stiffness or loss of support.

Note: The foot is designed to be used with a spectra socks and a foot cover.

BENCH ALIGNMENT (Figure 4)

- Fit foot with selected cover.
- Adjust to appropriate heel height (using the shoe).
- Introduce appropriate socket angles flexion/ extension and abduction/adduction.
- Divide the foot cover into 3 equal portions.
- The load line should fall at the junction of the posterior and the middle third portions as shown in figure 3.
- The 1/3 position is marked on the inside wall of the foot cover.

DYNAMIC ALIGNMENT

The heel stores energy after initial contact, slowly releasing it at midstance. The forward momentum generated by this action results in the toe being loaded for optimum energy release at terminal stance. The heel to toe action can be influenced by:

- A-P positioning of foot
- Dorsi-Plantarflexion
- Heel Stiffness

HEEL STIFFNESS

Heel too soft

Symptoms

- Foot will come to flat position too early (amputee feels he/she is sinking into a hole).
- Extra energy is required to climb up over the toe.
- Toe will feel too stiff
- Knee may hyperextend.

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Heel too hard

Symptoms

- Rapid heel to toe movement.
- At initial contact the amputee has poor control of his prosthesis.
- Minimal energy return feeling.
- Knee may become unstable.

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PROTECTIVE FILM ON PYRAMID

The protective film protects the pyramid from scratching. Remove the film when the fitting is complete.

SPECTRA SOCK

To protect the cover and avoid noise a Spectra sock is fitted over the foot module before installing the cover. The sock should be loose in the heel area.

SHOE HORN (Figure 3)

When removing and installing the cover use shoehorn to avoid damage to foot or cover.

MAINTAINANCE & CLEANING

Note: The device should always be used with spectra socks and foot covers to minimize dust and dirt from entering between carbon blades. It is important to fix the spectra sock around pyramid location in order to prevent noise issues coming from sand entering between the plates. The device is weather-proof but not resistant to corrosion. Therefore, the device should not come into contact with salt water or chlorinated water. The device is not designed to be used under extreme conditions like diving or jumping into water. If for some reason the foot components get wet parts should be dried with a lint free cloth. Should the device come into contact with salt water, chlorinated water, dust or dirt it must be rinsed with fresh water immediately.

DEBRIS BETWEEN BLADES

Noise could occur if sand or debris are present in between the upper and lower blades, same could happen between heel plate and lower blade. Use compressed air to remove debris in between the carbon parts.

WEATHERPROOF

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.


LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

COMPLIANCE

This component has been tested according to ISO 10328 standard to two million load cycles. Depending on the amputee's activity this corresponds to a duration of use of two to three years. We recommend carrying out regular yearly safety checks. In the standard mentioned, test

levels (P) are assigned to a certain maximal body masses (m in kg). In some cases, which are marked with, no test level is assigned to the product related maximal body mass. In these cases, the test loads have been adapted adequately on the basis of the specified load level.

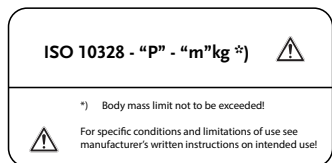ISO 10328 - label		
Category	Weight (Kg)	Label text
1	52	ISO 10328-P3-52-kg
2	59	ISO 10328-P3-59-kg
3	68	ISO 10328-P3-68-kg
4	77	ISO 10328-P4-77-kg
5	88	ISO 10328-P4-88-kg
6	100	ISO 10328-P5-100-kg
7	116	ISO 10328-P6-116-kg
8	130	ISO 10328-P6-130-kg
9	147	ISO 10328-P6-147-kg

WARRANTY

Össur provides a warranty of 36 months for this foot module and 6 months for the foot cover.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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