## Instructions for Use


---

### PRO-FLEX<sup>®</sup> LP




Product number:



PLP0xyz

PLPUxyz

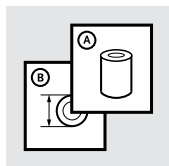		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	13
ES	Instrucciones para el uso	17
IT	Istruzioni per l'uso	21
NO	Bruksanvisning	25
DA	Brugsanvisning	29
SV	Bruksanvisning	33
EL	Οδηγίες χρήσης	37
FI	Käyttöohjeet	41
NL	Gebruiksaanwijzing	45
PT	Instruções de Utilização	49
PL	Instrukcja użytkowania	53
JA	取扱説明書	58
KO	사용 설명서	62
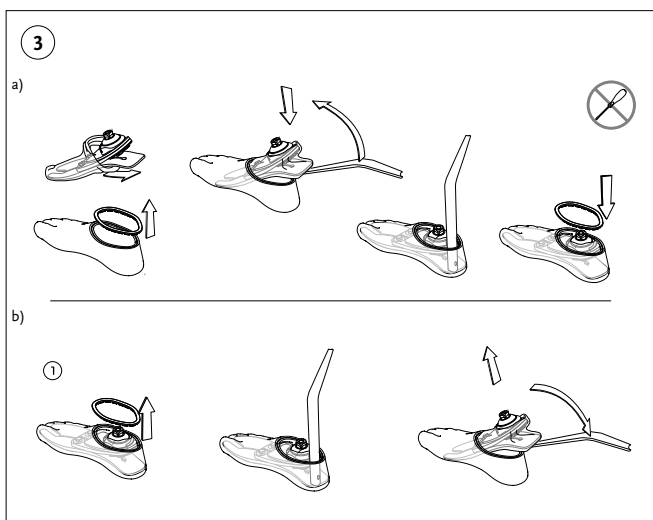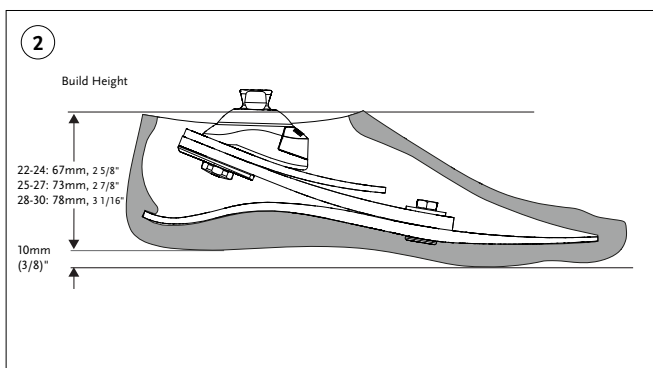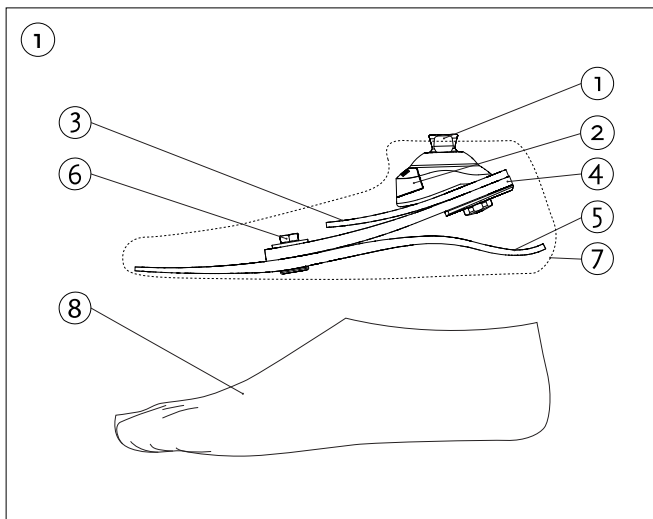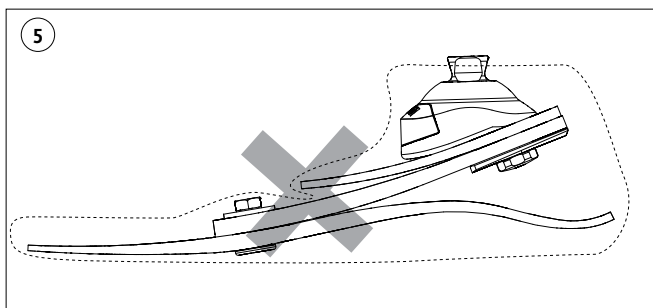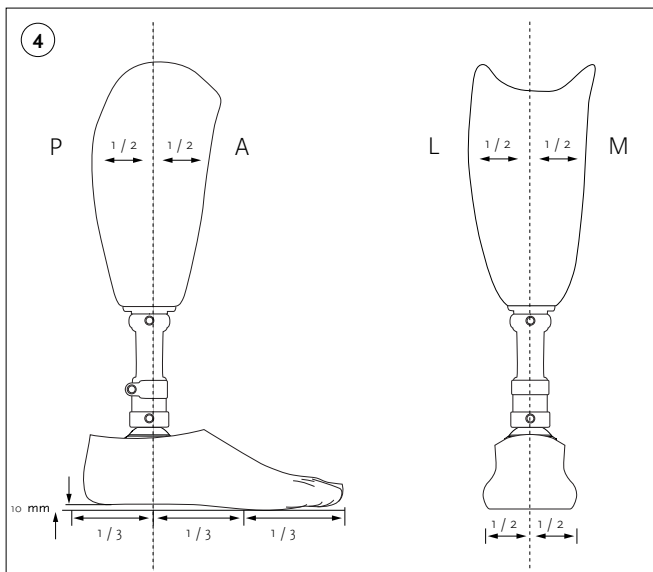
# ENGLISH

---

This manual is intended for use by a certified Prosthetist.

## PRODUCT DESCRIPTION

Pro-Flex LP is a prosthetic foot device with integrated male pyramid and three composite blades. It features a sandal toe and a full length composite blade. The bottom blade is secured together with heel bolts.

The device consists of the following components (**Figure 1**):

1. Male Pyramid Adapter
2. Pyramid cover
3. Top blade
4. Middle blade
5. Bottom blade
6. Bottom blade bolts
7. Spectra Sock
8. Footcover

## INDICATIONS FOR USE

The foot is a non-invasive, single patient, reusable prosthetic device. It is made for walking and everyday use of low to high active users.

The Pro-Flex LP is designed for single patient use, everyday activities and not for extreme sports. More details see “technical specifications” and “use environment”.

The Pro-Flex LP is tested and validated for exoskeletal prosthetic fittings of lower extremities amputations.

## TECHNICAL SPECIFICATIONS

Following specs are for a size 27 cat 5 foot:

- Product weight with footcover : 647 g / 1,42lbs
- Foot cover weight: 165 g / 0,36 lbs
- Build height: 73 mm / 2 7/8"
- Maximum user weight: 166 kg / 365 lbs

## CATEGORY SELECTION CHART

Please refer to the selection charts below to determine the appropriate stiffness required according to Össur recommendations.

Weight kg	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130	131-147	148-166
Weight lbs	99-115	116-130	131-150	151-170	171-194	195-221	222-256	257-275	288-324	325-365
Low Impact Level	1	1	2	3	4	5	6	7	8	9
Moderate Impact Level	1	2	3	4	5	6	7	8	9	N/A
High Impact Level	2	3	4	5	6	7	8	9	N/A	N/A

- Foot size 22–30

## SAFETY PRECAUTIONS

**Caution:** Statement that alerts the user to the possibility of a problem with the device associated with its use or misuse.

**Precaution:** Should be taken to avoid the hazard

**Note:** Possibility of a technical damage or malfunction of the device.

**Caution:** Make sure that the user knows if he/she feels or notices any change of function or alignment of the device, he/she should contact his healthcare practitioner. This noticeable function loss may be the result of reduced stiffness or loss of support.

**Note:** The foot is designed to be used with a spectra sock and a foot cover.

#### **BENCH ALIGNMENT (Figure 4)**

- Fit foot with selected cover.
- Adjust to appropriate heel height (using the shoe).
- Introduce appropriate socket angles flexion/ extension and abduction/adduction.
- Divide the foot cover into 3 equal portions.
- The load line should fall at the junction of the posterior and the middle third portions as shown in figure 3.
- The 1/3 position is marked on the inside wall of the foot cover.

#### **DYNAMIC ALIGNMENT**

The heel stores energy after initial contact, slowly releasing it at midstance. The forward momentum generated by this action results in the toe being loaded for optimum energy release at terminal stance. The heel to toe action can be influenced by:

- A-P positioning of foot
- Dorsi-Plantarflexion
- Heel Stiffness

#### **HEEL STIFFNESS**

##### **Heel too soft**

###### *Symptoms*

- Foot will come to flat position too early (amputee feels he/she is sinking into a hole).
- Extra energy is required to climb up over the toe.
- Toe will feel too stiff
- Knee may hyperextend.

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##### **Heel too hard**

###### *Symptoms*

- Rapid heel to toe movement.
- At initial contact the amputee has poor control of his prosthesis.
- Minimal energy return feeling.
- Knee may become unstable.

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#### **PROTECTIVE FILM ON PYRAMID**

The protective film protects the pyramid from scratching. Remove the film when the fitting is complete.

## **FOOT COVERS AND SPECTRA SOCKS (Figure 3)**

The foot is designed to be used with foot covers and spectra socks. When installing and removing the foot covers use shoehorn to avoid damage to the foot or the foot cover.

Please view the instructions for use for the foot covers before fitting them and ensure that the user knows how to use them.

It is important that the middle section of the spectra sock covers the toplade to make sure it doesn't intervene with the movement of the blades (**Figure 5**).

The thread of the sock should rest on top of foot module and not underneath the toe split.

Use a shoehorn to avoid damaging to the foot or the foot cover.

1. Installing – Make sure that the foot module is placed completely into the foot cover.
2. Removing the foot from the foot cover.

Note: Use spectra socks as shown in figure 1.

## **MAINTAINANCE & CLEANING**

**Note:** The device should always be used with spectra socks and foot covers to minimize dust and dirt from entering between carbon blades. It is important to fix the spectra sock around pyramid location in order to prevent noise issues coming from sand entering between the plates.

## **WEATHERPROOF**

A weatherproof device allows use in a wet and/or humid environment but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

## **DEBRIS BETWEEN BLADES**

Noise could occur if sand or debris are present in between the top and middle blades, same could happen between bottom blade and middle blade.

Use compressed air to remove debris in between the carbon parts.

## **USE ENVIRONMENT**

- Environmental operating temperature: -15 °C (+5 °F) to 50 °C (122 °F).
- Operating humidity: 10-95% relative humidity
- Shipping and storage temperature: -20 °C (-4 °F) to 70 °C (158 °F).

## **LIABILITY**

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

## **COMPLIANCE**

This component has been tested according to ISO 10328 standard to two million load cycles. Depending on the amputee's activity this corresponds to a duration of use of two to three years. We recommend carrying out regular yearly safety checks.

**ISO 10328 - "P" - "m"kg \*)**

\*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

In the standard mentioned, test levels (P) are assigned to a certain maximal body masses (m in kg). In some cases, which are marked with, no test level is assigned to the product related maximal body mass.

In these cases, the test loads have been adapted adequately on the basis of the specified load level.

ISO 10328 - label		
Category	Weight (Kg)	Label text
1	52	ISO 10328-P3-52-kg
2	59	ISO 10328-P3-59-kg
3	68	ISO 10328-P3-68-kg
4	77	ISO 10328-P4-77-kg
5	88	ISO 10328-P4-88-kg
6	100	ISO 10328-P5-100-kg
7	116	ISO 10328-P6-116-kg
8	130	ISO 10328-P6-130-kg
9	147	ISO 10328-P6-147-kg

**WARRANTY**

Össur provides a warranty of 36 months for this foot module and 6 months for the foot cover.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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