Instructions for Use


LOCKING KNEE[®]

Product Number: LKN12500

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	10
ES	Instrucciones para el uso	13
IT	Istruzioni per l'uso	16
DA	Brugsanvisning	19
SV	Bruksanvisning	22
NL	Gebruiksaanwijzing	25
PT	Instruções de Utilização	28
PL	Instrukcja użytkowania	31
JA	取扱説明書	34

1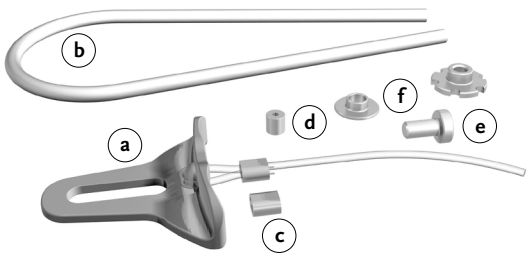

STABILITY AND SAFETY

The single axis joint has a stable and light frame built of aluminium-steel alloy with a proximal integrated male pyramid and a distal 30 mm tube clamp. The Össur locking knee offers a locking mechanism that can easily and effectively be unlocked by means of a release cord.

The Össur locking knee is designed for long-term use. Other than the biannual inspection and the use of lubricant 1500–025 or Centra Lube Oil 4401, the inner components do not require any additional maintenance or adjustment.

The joint will lock audibly by means of the locking mechanism built into the lower part of the joint and thus safeguards against accidental bending through the knee. By engaging the release cord, also under load, the mechanism is released, consequently enabling movement.

THE ÖSSUR LOCKING KNEE IS RECOMMENDED FOR:

- Low activity users that require a high degree of safety
- Bilateral transfemoral amputees
- Training prosthesis
- User with a body weight up to 125 kg (275 lbs).

TECHNICAL DATA

- Material: aluminium-steel alloy
- Reference height (from centre of knee): 26 mm
- Weight of knee: 248 g
- Weight limit: 125 kg (275 lbs)
- Maximum Flexion angle: 140 degrees

ASSEMBLY INSTRUCTIONS

The Össur locking knee is delivered ready-to-fit.

- Assemble the prosthesis: socket, knee joint, shin tube, adapters, foot module and shoe. The reference line corresponds to the load line.
- Align the tube adapter vertically, taking the heel height into account, to ensure optimal functioning of the knee joint.
- Tighten the pinch bolt of the tube clamp with 10 Nm and secure with Loctite 243.
- Determine the position of the socket according to the stump position of the user. Össur recommends 3°–5° (minimum) flexion and adduction. Take into consideration contractions and the resultant changes of the load line.
- Position the socket so that the reference line bisects the lateral side wall of the socket and falls through or not more than 5 mm behind the axis of rotation of the Össur locking knee.
- Determine the correct length of the prosthesis
- The release cord should run along the lateral or anterior side of the prosthesis.

TO ASSEMBLE THE RELEASE CORD ON THE ANTERIOR SIDE OF THE PROSTHESIS:

- Mould the screw insert into the socket at a desired location.
- Attach the knee joint to the socket.
- Fasten the handle assembly to the socket with the screw and collar

washer.

- Thread the release cord through the plastic tube and metal fastener.
- Thread the release cord through the central hole in the knee and again through the metal fastener.
- Shorten the release cord to the length that suits the user and make sure that the release cord is slightly tensed.
- Crimp the fastener by pliers or pinch the centre with a punch and mallet.
- Secure the plastic tube to the socket with a small amount of super glue or other means of preference

TO ASSEMBLE THE RELEASE CORD ON THE LATERAL SIDE OF THE PROSTHESIS, FOLLOW SAME INSTRUCTIONS AS FOR ANTERIOR SIDE EXCEPT:

- When running the cord through the central hole in the knee, use the crimp.
1. Handle assembly
 2. Plastic tube
 3. Metal fasteners
 4. Crimp
 5. Screw
 6. Screw Insert
 7. Collar washer
- Please ensure that the release cord is not impaired by the cosmetic cover, so the cord can move freely without friction.
 - Bench, statically and Dynamically align the prosthesis to the user

Further adjustments of the knee joint are not necessary.

MAINTENANCE

Biannual inspection

In order to guarantee perfect functioning, the Össur locking knee must be checked at least every 6 months with regards to the following:

- Assembly
- Secure locking of the locking mechanism
- Release of the locking mechanism
- Damage or wear and tear of the joint, release cord and screwed connections
- Any increase in weight of the patient (weight limit)
- Dirt in the knee joint

The joint must also be inspected when it has been subjected to extraordinary load (e. g. a fall).

Cleaning

Clean the joint with a soft cloth and some benzine. Do **NOT** use compressed air or aggressive cleansing agents.

Use of lubricants

Lubricate the joint at least every 6 months with general purpose oil (3-1). Flex the knee joint. Place few drops of oil on the joint. Move the knee several times and wipe off excessive oil with a soft cloth.

WARRANTY

If the Össur locking knee is used as prescribed and for the designated

purpose, Össur will provide a warranty of 12 months. The Össur locking knee was developed and produced in accordance with the ISO standard 9001 and features the CE mark.

COMBINATION

The Össur locking knee can be combined with all current parts complying with appropriate weight and activity load. We recommend use of the Össur locking knee together with a Flex-Foot Balance or the Flex-Foot Assure.

CAUTION! Avoid placing hands or fingers near moving joints.

LIABILITY

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

COMPLIANCE

This component has been tested according to ISO 10328 standard to 3 million load cycles. Depending on the amputee's activity this corresponds to a duration of use of 3 to 5 years. We recommend carrying out regular yearly safety checks

ISO 10328 - P6 - 125 kg ^{*)}

^{*)} Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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