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EN

### CAUTION

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and physical damage may result.

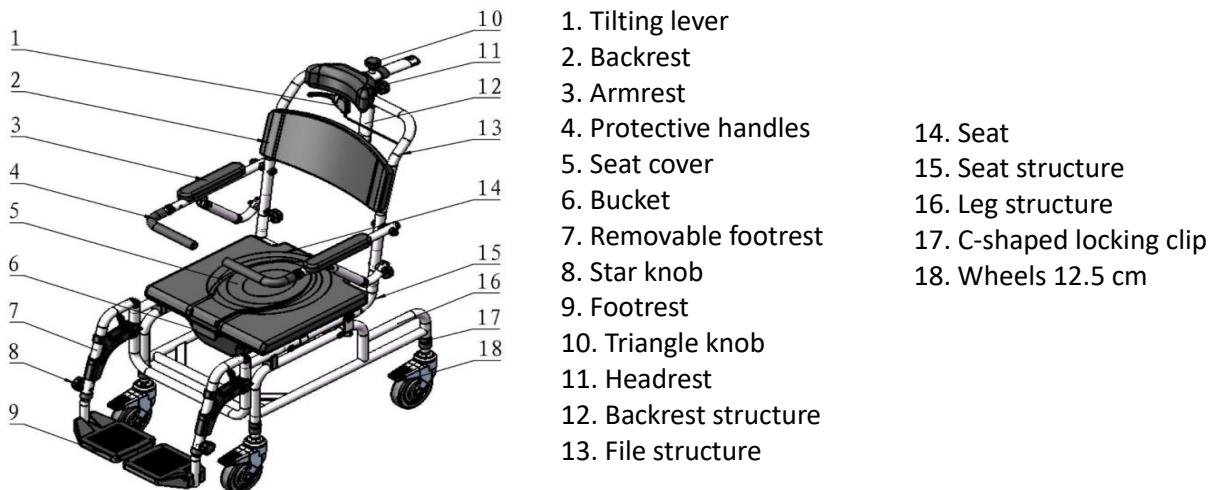
Consult your doctor or healthcare professional before using a medical device. He will recommend you the most appropriate use.

Distributor: this manual must be given to the users of this product.

### PRODUCT USE

Formentera Shower Chairs are technical aid devices for people with reduced mobility to enable them to take a shower without the risk of falling.

### PRODUCT DESCRIPTION

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 136 Kg.
- Do not use the shower chair on stairs or escalators.
- Only use the accessories and parts supplied by the manufacturer.
- Make sure that the knobs are tightened to the same level and the handle is securely fastened.
- Always sit on the seat when the shower chair is **stationary** and the brakes are locked. Serious injury to the user and/or damage to the wheelchair frame and/or wheels may result from improper use.
- Before using the Formentera shower chair, make sure :
  - That the chair is correctly assembled
  - That all chair components are properly secured
  - That the chair is free of any defects in appearance
- Make sure that the shower chair is locked before use.
- Make sure that all parts (wheels, jacks, cables, handles, brakes, headrest, guardrail, calf strap) are secure and in good working order.
- Do not make any adjustments to the product while it is in use.
- All wheels must always be in contact with the ground.
- Check that the brakes function properly before use.
- When the shower chair is in the stationary position, the brakes must be locked. Do not attempt to move the product if the brakes are locked.
- Make sure that the seat is clipped flat before use.
- Do not use the chair without the backrest.
- When sitting, do not try to reach objects by leaning forward.
- If any parts are damaged, broken or worn, contact your distributor.

## ASSEMBLY

- a) Insert the backrest frame into the seat frame as shown in fig. 1. Once correctly in place, lock it in

place using the star knobs.

- b) Install the four wheels and secure them with the C-clips (see image 2)
- c) Install the removable footrests on the seat frame (image 3).
- d) Insert the protective handles into the armrest tubes (fig. 4).
- e) Insert the frame of the headrest into the seat frame and lock it with the triangle knob (image 5).
- f) Install the headrest into the oval headrest structure and fix it with the triangle knob (picture 6).

Image 1

Image 2

Image 3

Image 4

Image 5

Image 6

## USE

- a) To adjust the distance and height of the headrest, unlock the triangular knob, adjust and then re-attach using the thumbwheel.
- b) To tilt the chair backwards :
  - First, lock all the brakes on each of the wheels to make sure that the shower chair stays firmly in place
  - Then press the tilt lever to tilt the chair backwards. The Formentera shower chair tilts from 97° to 125°.
- c) When installing the user, raise the protective handles and then put them back in the initial position.

## **Checklist :**

To keep the product in good condition, check before each use, and check regularly as follows :

Type of control	Method	Frequency
Cylinder operation	Adjustment	Once a week
Frame stability	Adjustment	Once a month
Mounting brackets (screws, knobs)	Adjustment	Once a week
Brake control	Adjustment	Before using
Checking the guardrail	Adjustment	Once a month
Headrest check	Adjustment	Once a month
Backrest check	Adjustment	Once a month
Checking the calf straps	Adjustment	Once a month

## **CARE AND MAINTENANCE**

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- If there is a malfunction, do not repair the product yourself. Consult an authorized distributor.

## **CHANGE OF OWNERSHIP**

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## **RECYCLING**

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling by yourself, ask your local waste management company for disposal guidelines.

## **STORAGE**

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your products.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## TECHNICAL SPECIFICATIONS

Overall dimensions	Width 55.5 x Depth 101 cm
Adjustable Height	118 to 123 cm
Seat Width	Width 43.5 cm x depth 43.5 cm x height 55.5 to 60.6 cm
Width between armrests	54 cm
Structure	Aluminum corrosion resistant
Weight	14.5 kg
Wheel diameter	12.5 cm
Maximal load	136 kg

## PERIOD OF USE

The life expectancy of this product is 3 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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