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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Moovy toilet/shower and transfer chair is a medical device. It is a technical toilet aid for the elderly, people with reduced mobility, disabled, injured or who can easily lose their balance.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 120 kg
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Before each use of the product, the brakes and wheels should be checked.
- Check the stability of the chair before use.
- Only use on level ground.
- When stationery, apply the brakes on the wheels.
- Ensure the brakes are released before the chair is moved again.
- Always ensure that the device is stationary and the brakes are applied when transferring to and from the device.
- Do not lean out of the device.
- Check the wheels bearings the brakes
- When the user sits or stands up, keep the feet on the floor and not on the footrests.
- Please do not adjust the height when the user is sitting.
- In case of damaged, broken, worn or missing parts, contact the dealer.


MOUNTING & USE

MOUNTING :


- a) Place the rotatory shafts into the tubes before screwing the M12 nut. Make sure the shafts can be flexibly rotate without being loose, as illustrated on the right :

b) Place the handle part into the right shaft, as illustrated on the left :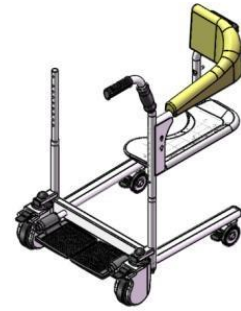



c) Place the handle part into the left shaft, as illustrated on the right: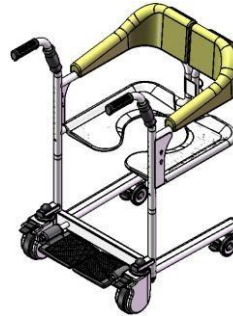

d) Adjust its height by rotating the handle to fit their hole sites into that of the rotatory shafts, then plug into the lock catches. The operation is illustrated on the left:

e) For placing a bedpan, fold the handles on both sides and place the bedpan holder in the hole site, then place the bedpan from the rear, as illustrated on the right :

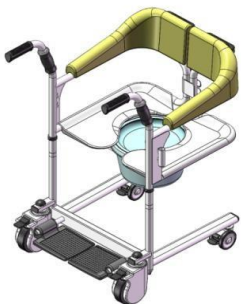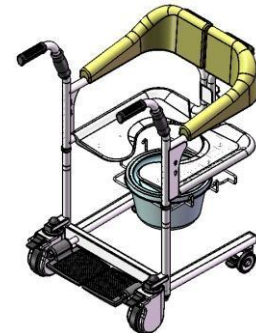f) Fit on the small tube for infusion support. The tube is detachable and changeable to left or right side. As illustrated on the left :

Please follow all the instructions stated in this user manual when using the Moovy toilet/shower and transfer chair. This is for your security.

Notes : For safe and smooth using, the bedpan and bedpan holder should be installed only when needed.

USE :

a) The patient should sit up straight when the chair is unfolded in front of the patient. Lift the patient's feet to step on the rest pedals. Make sure its brakes are locked until the patient is seated on the chair.

b) When the patient is leaning to the left and to the right, turn the correlative side of the chair to fit the patient into it. After the patient is seated, fold the machine from its back, and lock the auto buckle. Then lock the safety buckle.

c) Unlock the brakes when the patient is seated with double buckles locked. Now take hold of the front handle to move the patient.


d) Unlock the double buckles and unfold the machine after the patient is moved to the target place.

ADJUSTING THE HEIGHT :

a) Take down the lock catch to adjust the height. There are regulation ports on the sticks.

b) After adjusted to desirable height, put the lock catch back to its place.

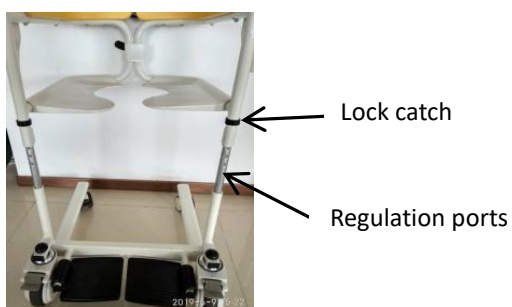CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product (all wheels, seats, and fixations) with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The chair must be completely reviewed every year, with compulsory lubrication.
- It is recommended that the device is serviced annually.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

| Check the item | Any time | Weekly | Monthly | Yearly |
|----------------------|----------|--------|---------|--------|
| Four wheels | X | | | |
| Belt of protection | X | | | |
| Cleanliness | X | | | |
| Screws and fasteners | | X | | |
| Lifting motor column | | | X | |
| Frame stability | | | X | |
| State of the seat | | | | X |
| Brakes | | | X | |

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 3 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

| | |
|--------------------------|--|
| Overall dimensions | Width. 53,5 cm x Depth 70 cm x Height 76 / 94 cm |
| Clearance height | 12 cm |
| Seat dimensions | Width 42,5 cm x Depth 42 x Height 42 / 60 cm |
| Weight | 21 kg |
| Maximum supported weight | 120 kg |

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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