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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The 4-wheels aluminium rollator is a walking aid for people with reduced mobility. This rollator is designed for indoor and outdoor use and folds up for easy storage or transport.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 135 Kg.
- Do not exceed the maximum load limit in the basket (5kg). Only transport loads in the basket. Do not hang heavy objects on the rollator as this could affect its stability.
- Make sure that all parts (wheels, cables, handles and knobs) are secure and in good working order.
- Do not use the rollator on stairs or escalators.
- Only use accessories and parts supplied by the manufacturer.
- The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.
- Do not use the rollator to walk backwards.
- Do not make any adjustments to the rollator while it is in use.
- Check that the brakes work properly before use.
- All wheels must always be in contact with the ground.
- When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.
- Be sure to put your hands on each handle so that the rollator can be in the parallel force position.
- Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a

significant risk of falling or injury.

- Do not use the walker on a wet, icy, or snowy slope.
- If any parts are damaged, broken or worn, contact the dealer.

MOUNTING & USE

1. Remove the rollator and all its components from the packaging.
2. Open the rollator and push the seat down, ensure the side hinges are in the open and locked position.
3. Install the backrest by inserting each end into the slots provided. The "click" indicates that the backrest is correctly installed.
4. Adjust the height of the handles. The heights must be the same on the right and left. Lock the tubes by tightening the knobs on both sides. Make sure that the cables remain outside the structure and that the tubes are not crushed or bent.
5. The correct height for the handles is when you stand upright with your hands on the handles without bending your wrists.
6. To slow down, squeeze the brake handles slightly with your fingers. Squeeze the brake handles simultaneously on the right and left sides.
7. To lock the brakes, e.g. to stop the rollator, push both brake handles firmly downwards until they lock by themselves.
8. To unlock the brakes, pull up both brake handles.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat				X
Brakes	X			

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width. 60 x Depth. 74 x Height. 82 / 97.5 cm
Seat dimensions	Width. 35 x Depth. 33 x Height. 58,5 cm
Width between the handles	47,4 cm
Maximum bag load	5 kg
Maximum supported weight	135 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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