ROLLATOR NEO XL

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESCRIPTION

The Neo XL Rollators are walking aids for people with reduced mobility. These products are intended for use indoors and outdoors and can be folded up for easy storage or transport.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 181 Kg.

Do not exceed the maximum load limit in the basket (5kg). Only transport loads in the basket. Do not hang heavy objects on the rollator as this could affect its stability.

Make sure that all parts (wheels, cables, and handles) are secure and in good working order.

Make sure that the wheels are tight and at the same height.

Do not use the rollator on stairs or escalators.

Only use accessories and parts supplied by the manufacturer.

The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.

Do not use the rollator to walk backwards.

Do not make any adjustments to the rollator while it is in use.

Check that the brakes work properly before use.

During use, lock the parking brakes for all actions.

All wheels must always be in contact with the ground.

When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.

Be sure to put your hands on each handle so that the rollator can be in the parallel force position.

Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

Do not use the walker on a wet, icy, or snowy slope.

If any parts are damaged, broken or worn, contact the distributor.

MOUNTING

Remove the Neo XL rollator from its carton and check that all components are present. Remove all packaging and protective covers. Make sure all components are in good condition before assembly.

- a. Unfold the rollator by unfolding the frame as shown in Figure 1. Place the frame on a flat surface. Gently press down on the seat to fully open the frame.
- b. Insert the backrest at the back of the frame as shown in picture 2. To insert the folder, press the buttons at the ends of the folder. Once in place, pull the backrest very lightly to ensure that it is correctly fixed.
- c. Lay the Neo XL rollator on the floor so that it rests on the back of the frame. You will squeeze in front of the bag. (Image 3)
- d. Insert the front wheels into the frame. The wheels are properly secured when you hear a "click" sound. (Image 4)
- e. Turn the rollator over on the floor so that it rests this time on the side of the front wheels. (Figure 5)
- f. Insert the rear wheels (marked with an "R") into the rollator frame. The wheels are securely attached when you hear an audible "click". (Figure 6)
- g. Place the rollator back on the floor on these 4 wheels. Make sure that all wheels are positioned at the

same height and that the rollator is not wobbly. (Figure 7)

- h. Insert the handles into the frame, making sure that the outside the rollator frame. (Figure 8)

- i. Adjust the height of the handles as required by pressing on the handle tubes (image 9). Make sure that both at the same height.

image 9

cables are all

the buttons
handles are

NOTE: Make sure that all four wheels are positioned at the same height before each use.

USE

OPEN THE ROLLATOR

- Hold the handles and press down on the seat until all four wheels are aligned and the seat is completely flat.
- Before using the rollator, make sure that all four wheels are in contact with the floor and that the rollator is stable in the open position.

FOLDING THE ROLLATOR

- Lift the seat and lift the seat support bar until the rollator closes as shown in Figure 2.
- For more compact storage, pull the backrest up.

image 10

ADJUST THE HEIGHT OF THE HANDLES

Note: Do not remove the inner tube from the outer tube when adjusting the height of the handles.

image 1

image 2

image 5

image 6

image 3

image 4

image 7

image 8

- For the rollator to be at the correct height, stand behind the rollator with your arms at your side. Then adjust the height of the handles to the height of the user's wrists. (picture 10) .

WALKING WITH THE ROLLATOR

- When using, stand upright behind the rollator, do not bend your back.
- Adjust the height of the handles if the position of use is not correct.

SITTING POSITION

When using the rollator as a chair, ensure that all safety precautions are followed, especially :

- Make sure that the parking brakes are locked throughout the sitting period and when changing positions from standing to sitting and from sitting to standing. Do not use as a seat unless the backrest is in the
- Do not sit down when the rollator is on an inclined or uneven surface.
- Do not exceed the maximum weight
- This rollator is not a wheelchair. It is not designed to carry a person sitting in it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use.

USE OF THE BRAKES

- To slow down, pull both handles upwards.
- To activate the parking brakes (lock), push both handles firmly down until they engage. A distinct click indicates that the brakes are locked. Pull both handles upwards to release the parking brakes (unlock).

BRAKE ADJUSTMENT

There are two brake cable tensioners on each side of the Rollator, one in front of the brake handle and the second is positioned on the brake caliper near the rear wheel. Adjustments can be made to either tensioner.

- To adjust near the handle, loosen the knob (you may need a tool to do this), and turn the adjusting screw counterclockwise to tighten the brake cable and then tighten the knob again. - For adjustment at the rear caliper, loosen the adjustment knob, (you may need a tool to perform this operation), and turn the adjustment screw counterclockwise to tighten the brake cable, and then tighten the adjustment knob again.

For correct adjustment, the brake mechanism should not touch the wheel, but the distance between the brake shoe and the wheel should not be more than 3 to 4mm.

- Caution: After adjusting the brakes, you must test them before using the rollator.
- If, despite the adjustment, the brakes still do not work, do not use the rollator.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.

- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

• Checklist :

To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat				X
Brakes	X			

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your rollators.

- In a dry and temperate place
- Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the rollator elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 73 cm x Depth 73/78 cm x Height 83/98 cm
Height adjustable from	83 to 98 cm in 5 positions
Seat	Width 42 cm x height 53/57 cm x depth 39.5 cm
Width between armrests	54 cm
Maximum load	181 kg
Aluminium structure	
Weight	8.8 kg with footrest
Wheel diameter	20.5 cm

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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