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## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

## PRODUCT USE

The Neo Global Transfer Rollator is a walking aid for people with reduced mobility. These rollators are designed for indoor and outdoor use and fold up for easy storage or transport.

## DESCRIPTION

## SAFETY PRECAUTIONS

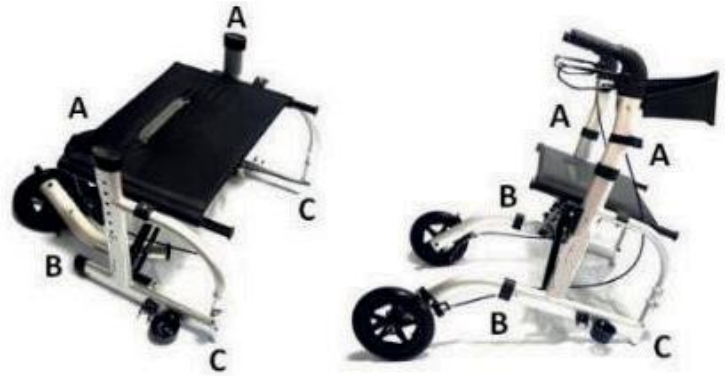
Failure to observe the precautions for use may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 136 Kg.
- Do not exceed the maximum load limit in the basket (5kg). Loads must only be transported in the basket. Do not hang heavy objects from the rollator as this could affect its stability.
- Make sure that all parts (wheels, cables and handles) are in good working order and securely fastened.
- Make sure that the wheels are tight and at the same height.
- Do not use the rollator on stairs or escalators.
- Use only accessories and parts supplied by the manufacturer.
- Do not use the rollator for walking backwards.
- Do not make any adjustments to the rollator while it is in use.
- Check that the brakes work properly before use.
- During use, lock the parking brakes for all actions.
- All wheels must always be in contact with the ground.
- When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.
- Be sure to put your hands on each handle so that the rollator can be in the parallel force position.
- Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of

falling or injury.

- Do not use the walker on a wet, icy, or snowy slope.
- If any parts are damaged, broken or worn, contact the dealer.

## ASSEMBLY

1. Place the Neo Global Transfer Rollator on a flat surface for assembly. Open the rollator as shown in the picture, the seat should be fully open in the horizontal position.

2. Insert the two handle tubes into sections A, B, C and D of the rollator. Make sure that both handles are positioned at the same height.

3. Then insert the two rear wheels in sections B of the image above. Make sure that they are properly secured.

4. Insert the two front wheels in sections C of the image on the right. Secure the front wheels with the triangle-shaped knob.

5. Finally, install the removable footrests. Be sure to install the footrests at the same height on each side of the rollator.

### Handle adjustment

Note : Do not remove the inner tube from the outer tube when adjusting the height of the handles.

For the rollator to be at the correct height, stand behind the rollator with your arms at your side. Then adjust the height of the handles to the height of the user's wrists.

### Adjustable handle height

The handles are height adjustable from 84.5 to 97.5 cm from the floor in 6 positions. To ensure that the rollator is at the right height, stand behind the rollator with your arms at your sides.

Then adjust the height of the handles to the height of the user's wrists. In case of doubt, consult your dealer.

## USE OF THE ROLLATOR

### Braking

- To brake, pull the brake handles upwards.
- To set the parking brakes, push both handles firmly down until they lock into position. Attention, to be in parking function, both handles must be locked at the same time.
- Lift both handles to unlock the parking brakes.
- If the brakes are in operation but the rollator wheels are still moving, discontinue use immediately. Do not use the rollator until the brakes have been checked and properly adjusted.

## CARE AND MAINTENANCE

Failure to care for and maintain your rollator can result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, soft, dry cloth.
- Make sure there is no accumulation of dirt or debris around the axle or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts.
- Check that the brakes work before each use.
- Check the parking brakes by pushing the handles into the locked position and then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

## CHANGE OF OWNERSHIP

The rollator can be reused. It must first be cleaned and disinfected in accordance with hygiene regulations. When transferring the rollator, remember to give all the necessary technical documents to the new user. The rollator must first be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to comply with the storage conditions may lead to deterioration of the product and thus the risk of serious injury.

- Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Observe the conditions for storing and stowing your rollators :
  - In a dry and temperate place
  - Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
  - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
  - All components must be stored empty (do not place heavy objects on the rollator elements).

## TECHNICAL SPECIFICATIONS

Overall dimensions	Width 60.5 x Depth 74 cm (104 cm with footrest)
Height adjustable	84.5 to 97.5 cm in 6 positions
Seat	Width 41 cm x height 55 cm x depth 31 cm
Width between armrests	46.5 cm
Maximum load	136 kg
Aluminium structure	
Weight	9.5kg (including footrest)
Wheel diameter	20 cm

## OPERATING TIME

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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