BARRE D'APPUI A VENTOUSES STRONG

REF 815112

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Description :

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USER MANUAL

Description:

This handle should only be used as a balance aid. It cannot support your total weight and is not designed to support such a pull.

Before each use, check the adhesion of the two suction cups by pulling slightly without support. If one or both suction cups are not adhering to the wall, you must remove and reposition them. Check the tightness of the suction cups every time you use them.

Note: For maximum safety this product must be used correctly. Improper use may cause the handle to become unstable or detached from its base.

Instructions for use:

Clean and moisten the surface before installing the handle. The levers must be lifted. Keep the suction cups fully depressed so that they cover the entire surface. Lower the levers to secure the handle. Check the grip, repeat if necessary. To remove the handle, raise the levers and remove the handle while pulling the suction cup tab slightly. Check the grip before each use. If necessary, clean the suction cups and reinstall the handle following the previous instructions. Disassemble and reassemble every week.

The suction cups must be placed on a smooth, flat, clean and non-porous surface. Placing the suction cups on cracks or joints between tiles prevents the handle from being fixed properly;

Caution: apply the product to a smooth tiled surface, ensuring that the surface can withstand the system (do not use on poorly bonded tiles, textured, porous or rough tiles, painted or papered surfaces).

Maximum weight supported by the bar 40 kg

Warning:

- Make sure the handle is properly attached before use, check it, and replace it if necessary.
- THE PULL MUST REMAIN PERPENDICULAR TO THE HANDLE AS SHOWN IN THE PHOTO.
- Placing the handle on a surface that is too wet can cause it to slip and fall off.
- Check the good condition of each suction cup before installation: The suction cup must not show any abnormal deformation (bulging, cracking, etc.).
- The suction cup must not show any abnormal deformation (bulge, cut, swelling). It must be flat, smooth and slightly smooth and slightly flat.
- Before each use, check the adherence of the 2 suction cups by pulling slightly without leaning. If one or both suction cups are not adhering to the wall, you must remove and reposition them. Check the tightness of the suction cups every time you use them.
- The levers must be resistant when operated. If there is no resistance means that the suction cup is not operational.
- The suction cup must protrude at least 1 mm all around the plastic part. If the suction cup does not protrude (even very partially), it will easily come off = potential danger.
- Do not overestimate the hold of the product and use it sensibly.
- This system has the advantage of a quick installation but implies a quality installation on a quality support.quality substrate.
- Remember to test the hold firmly before you actually stand on the product.
- This product is not intended for people with disabilities or physical problems. It should not be used as a support ramp.

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