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# INSTRUCTIONS MANUAL EN

The annotations and illustrations in this manual might be slightly different with the real parts due to quality improvement or changing design. Please in kind prevail.

Contact with your dealer if there is any ambiguity or question.

Improper use of any vehicle may lead to injury. Unsafe driving could harm yourselves and others.

## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a mobility aid. He or she will advise and inform you about the most appropriate use in your case.

Distributor: this leaflet must be given to the users of this product.

## PRODUCT DESTINATION

Motorlight Folding Chairs are medical devices. It is a technical aid designed to comfortably transport people with reduced mobility. This electric wheelchairs are designed to transport 1 person only. They are intended for disabled and elderly people (less than 125 kg) and serve as a walking vehicle. They are suitable for outdoor use over a medium distance. This series of electric wheelchairs are powered by battery, driven by motor. Users control direction and adjust speed by joystick controller. It is suitable for application of low speed, good road condition and small slope.

## PRODUCT DESCRIPTION

### Main Structure

This series of electric wheelchairs are mainly composed of two parts:

- 1) Frame Part: frame, wheel, footrest, armrest, backrest etc.
- 2) Electrical Part: controller, motor, battery.

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 125 Kg.
- Make sure that all parts are well fixed and in good working order.
- Only use the accessories and parts supplied by the manufacturer.

- Check the correct operation of the brakes before use.
- During use, lock the brakes for all actions.
- All wheels must always be in contact with the ground.
- When the product is in a stationary position, the brakes must be locked. Do not attempt to move the product if the brakes are locked.
- Make sure that the footrests are properly engaged.
- Make sure the removal pin is securely engaged in the rear wheels and frame of the chair.
- Do not stand on the footrests of the product as there is a risk of the product tipping over.
- The maximum speed of this electric wheelchair is not more than 7 km/h

### General Driving Cautions

- Please keep your hands on the armrest to control the wheelchair.
- Please practice in parks or other safe open places until you can use the wheelchair skillfully.
- Fully practice driving in a safe place, to master principle of moving forward, stopping and turning circles.
- Before driving to the road, please be accompanied by caregivers and confirm it is safe.
- Please strictly follow traffic rules of the pedestrian, do not regard yourself as vehicle driver.
- Please drive on the sidewalk and the zebra crossing. Do not drive on vehicle lane.
- Steadily drive, to avoid driving in 'Z' line or sharp turn.
- Please keep pressure of pneumatic tire properly. The abnormal tire pressure may cause unsteady driving or excessive lost of current.
- Do not run on the motorways.
- Accompanied by caregivers or avoid driving in these conditions :
  - Drive in bad weather, such as rainy day, heavy fog, strong wind, snow, etc.
  - In case of wheelchair is wet out, wipe the water immediately.
  - Drive on the bad road condition, such as muddy, trail, sand road, gravel, etc.
  - Drive on crowded road.
  - Drive on no fence side ditch, pond, etc.
  - Across the railway.
- When you have to across railway, pause at turnoff to confirm it is safe, and make sure that the tires will not be stuck by railway.
- The electric wheelchair is only for personal use, do not carry people or goods, and do not for a traction purposes.
- Put on the seat belt before each use

### Precautions for uphill and downhill

- Avoid driving on following places: steep hills, tilt places, high steps, channels etc.
- Avoid driving on steep slopes; the slope range should be less than 9 degrees. Please carefully operate the controller when drive on slopes.
- Keep moving forward during uphill or downhill.
- Slow down speed during downhill.
- Avoid driving sideways on the steep.
- Forbid driving on the stairs place and avoid crossing high steps.
- Avoid crossing wide ditches.
- During crossing ditches, keep 90° angels between tires and the ditches.

Do not set the wheelchair to manual mode during uphill and downhill

When the wheelchair is malfunction at traffic crossing, please immediately ask passerby for help. And set wheelchair to manual mode, then push wheelchair to leave the scene, or user to get away from the site to a safe place immediately.

### Precautions for the Caregivers


Caregivers should confirm that the users' feet are on the footrests' proper position and ensure clothes do not attach to the wheels.

Caregivers should push the wheelchair to move forward to keep safe on steep slope or long slope.  
If any parts are damaged, broken, worn, or missing, contact your distributor.



## MOUNTING AND USE

### Fold Wheelchair

Step 1: Switch on in upward direction

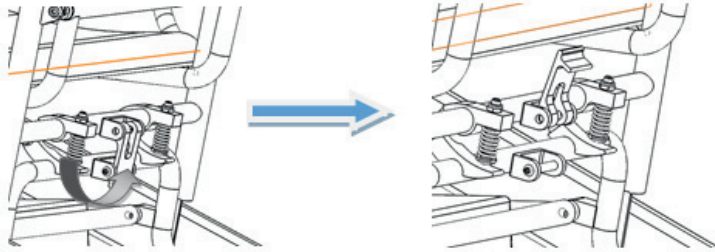
Step 2: Compress the backrest tube to be completely folded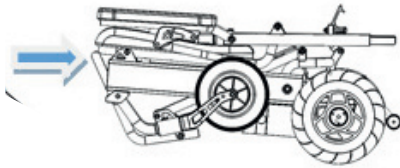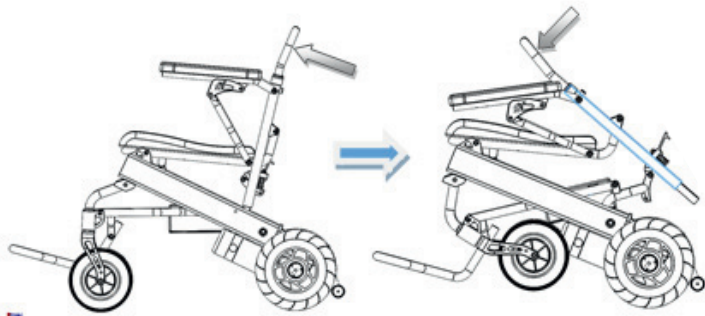


### Armrest Usage

Step 1: Turn the knob switch down

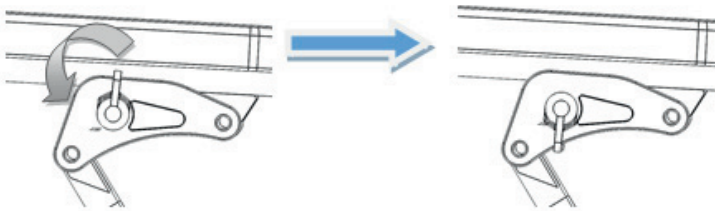
Step 2: Uplift the armrest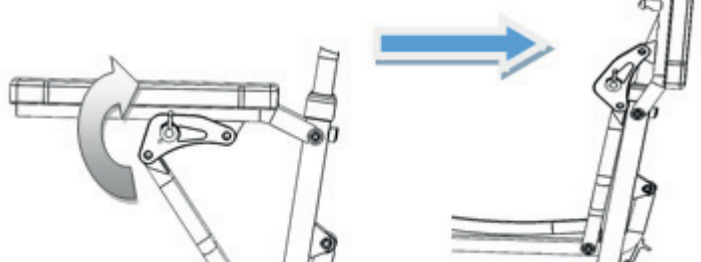

### Adjustment of Joystick

Put the tube below the controller into the tube below armrest. Adjust the controller to perfect position by screwing the nut counterclockwise (picture 3). Screw tight the nut when adjustment is done.

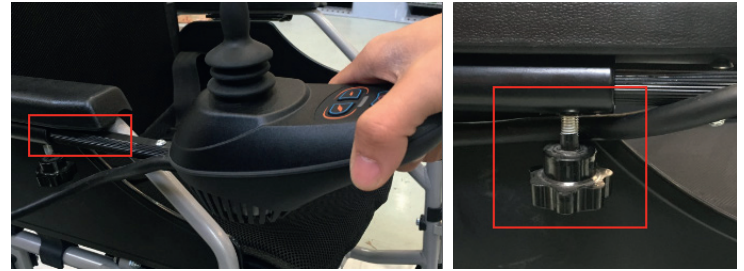### Switch between Manual Mode and Electric Mode

When changing from manual mode to electric mode, please confirm both side clutches are inside the grooves. Any side clutch is not fully into groove, may cause danger of rollover.

**Manual mode and power mode switch:** turn off power and stop the wheelchair, spin the adjusting levers.

Do not make wheelchair into manual mode when user is driving on slopes

### Battery and Charger

#### BATTERY

The batteries are assembled already. It's foldable without moving the batteries.

#### CHARGER REQUIREMENTS

The charger is used for battery charging. During electric wheelchair is charging, do not use it.

#### CHARGER TECHNICAL DATA

Input voltage: AC220V  $\pm$ 10%

Output voltage: 24V/DC

Output current: 3A.

#### USAGE OF CHARGER

In order to charge the battery, connect the charger with power supply and battery box's plug. Do as following instructions to complete charging process :

- Step 1: Make sure the charger groove is not blocked.
- Step 2: Make sure the electric wheelchair is power off.
- Step 3: Unplug the plugs that connect the battery box and controller.
- Step 4: Connect the charger's output plug to the battery box's power plug.
- Step 5: Connect the charger's main plug to the power supply and the red light will be alight. Fully charging needs 8 to 10 hours, do not overcharge more than 24 hours.
- Step 6: As to prevent cut short battery life, please charge the battery at least once per month when the wheelchair is not being used.

Do not stop charging until charge process is finished. Repeat using the battery, which is not fully charged, will shorten the battery life, so the battery should be fully charged as much as possible.

When battery is fully charged, the power indicator will turn to green. Do not stop charging before fully charged. After finish charging, turn off the power supply, otherwise the battery will discharge slowly. Do not charge more than 24 hours. Over-charging is dangerous

 Users should follow the following rules to avoid charging dangers:  
The electric wheelchair does not include the charger, please use national standard charger that the output voltage is 24V, 3A. It should be well ventilated when charging. Do not expose the wheelchair to sunlight and humid environment.

 The charging environment temperature range is from 10 to 50°C. If it is out of the environment temperature range, the battery is unable to function well, and can easily make battery damaged. It is normal for the fan making a sound during charging. It is for cooling the charger, please do not worry about it.

 Prevent liquid go into the charger during charging. Moreover, do not place charger on the flammable items, such as fuel, footrest or seat cushion.

**THE USAGE AND MAINTENANCE OF BATT**

Wrong operation of replacing battery may cause danger of explosion. Only the same or recommended type of battery is suitable for replacement. In addition, please make sure the battery poles are correct. Key points for prolong the battery life: charge frequently, to keep battery power full. It is better to fully charge the battery if the wheelchair is not being used. If stop using for a long time, it is better to charge twice per month.

 Please stay away from the flames when charging the battery. The flames may cause battery fire or explosion. Charging makes hydrogen. Do not smoke while charging. Do not unplug the power supply when the socket or your hands are wet, it may cause electric shock. In case of unpredictable accident will happen and hurt the user, do not use or sit on the electric wheelchair when charging.

**System Diagnose**

When the indicator LED lights are blinking, it means the wheelchair has abnormalities. The abnormalities may occur in following parts: motors, brakes, battery, wire connections, etc. Through the product's inner information consultation, the property of the abnormal condition can be detected by the diagnosis signal. The abnormal situation can be detected without other service tools.


**AUDIO SIGNAL INDICATION**

Description of LED light	The meaning of LED light	Explanation & Solution
All LED lights are unlit with no sound	The power is off, wheelchair in standby or sleep mode. Power is poor contacted. Fuse is tripped or burned out.	
All LED lights are lit	The power is turned on, and self diagnose is passed, the electric wheelchair can work well.	Less LED lights lit, less battery power remaining.
The leftmost red LED light is lit	The battery power is extremely inadequate.	To charge immediately. Or battery is malfunction, and is not able to charge.
Two short beeps with five indicator lights blink twice	The left side motor is malfunction.	The left side motor is poor connected or the wire is disconnected.
Four short beeps with five indicator lights blink four times	The right side motor is malfunction.	The right side motor is poor connected or the wire is disconnected.

Six short beeps with five indicator lights blink six times	Controller is in over-current protection status.	Check the brakes, and check if the motor drive mechanism is stuck. Check the current by ammeter, if it is not excessive current, maybe the controller is malfunction.
Seven short beeps with five indicator lights blink seven times	Joystick is malfunction	Joystick does not reset, or the connector is loose.
Eight short beeps with five indicator lights blink eight times	Controller is malfunction	Please consult your dealer for maintenance.
Nine short beeps with five indicator lights blink nine times	Controller is malfunction.	Please consult service center for maintenance.

**Controller**

**CONTROLLER PANEL**

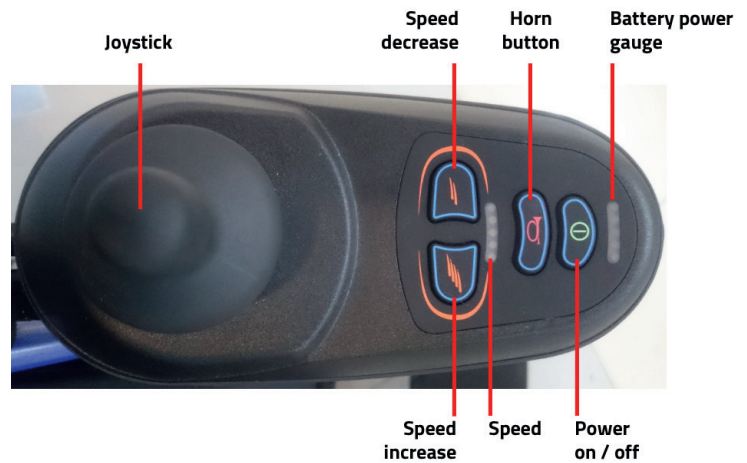**CONTROLLER USAGE**

**Power Switch**  
Press this button, the battery power gauge lights will turn on from left to right. Press again, all the LED lights are off.

In some emergency, you can directly turn off power by press power button.

**Sleep mode**

If the joystick has no operation more than 20 minutes, the power turn off automatically, and the system in the sleep mode, system will be woken up from sleep mode by press power button.

**Speed adjustment**  
According to user's habits and the circumstances, the wheelchair driving speed is adjustable.

Adjust speed by press decrease button or increase button. Speed is divided into five sections, ranged by 20%, 40%, 60%, 80%, and 100% of the max speed.  
First gear of speed: 20% of the max speed.  
Fifth gear of speed: 100% of the max speed.

**Horn Button**  
Press the horn button, the horn will sound until you release the button.

### Battery Power Gauge

After turn on the power, the battery power gauge is working. The battery power gauge also shows the battery remaining power capacity.

As the picture shown, the battery power is full.

When there is only the red or yellow LED light lit, the battery should be charged. And for a long distance driving, the battery should be fully charged. If only the red LED light is lit, the battery power is extremely inadequate, users need to charge battery as soon as possible.

### Use of joystick

The wheelchair moving direction is controlled by joystick. The moving extent of joystick also controls the moving speed.

Please turn on or turn off the power when the joystick is in the middle position. Otherwise, the controller will let out wrong operation signal.

Release the joystick to middle and this mistake will stop. If the mistake is continuing, perhaps this part is malfunction, do not use it and contact with your dealer for maintenance.

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

Maintenance of wheelchair includes clean wheelchair, check wheel and battery, and charge battery. For further maintenance, please contact with your dealer. Our suggestion is checking the wheelchair every half year, annually overhaul.

### Clean and Battery Maintenance

#### CLEAN WHEELCHAIR

Please clean the wheelchair regularly.

Clean the parts, which are frequently touched with user's body (such as set cushion, armrest, controller) by a clean and slight wet cloth. Do not use organic solvents to clean.

If it is a patient user, the wheelchair should be cleaned once per week. If the wheelchair is used by an infectious victim, it should be cleaned and disinfected by disinfectants.

#### WHEELS

Check the tires' air pressure and their wear condition regularly. When the tire tread pattern depth reduces to 1mm, please replace new tire.

#### BATTERY

To make sure that the battery is fully charged frequently. In order to prolong battery life, we strongly recommend users do not to charge until battery run out of power totally.

#### WHEELCHAIR MALFUNCTION AND CHECKING

When malfunctions happen during the wheelchair is working, please turn off the power before check.

Symptom: completely lose power, and all the LED lights on the controller panel are off.

#### Check Step :

Step 1: To check if the controller's plug is loose.

Step 2: To check if the connection of controller's plug and battery box is loose. Please reinsert the plug connector (hold the plug when pull out the plug. Do not pull the wire to avoid unnecessary damage to the wire line). After above checking, if the wheelchair is still not able to restore the power, or if users have any question for above checking, please connect with your dealer.

The controller has a diagnosing system to monitor the controller and motor. Any malfunction of these parts is indicated by the controller. For more details, please kindly refer to chapter of audio signal indication.

### Maintenance Checking

To ensure the optimal functioning of the product, please check as follows: **1:** Check that the joystick is attached on the right or left side and

the condition of the wires from the wheel casings to the joystick (once a week). **2:** Check the freewheel function and the engagement. **3:** Check battery (once a month). **4:** Check folding and unfolding, product **4:** Check folding and unfolding, product clicked in unfolded position (once a month). **5:** Check the speed of the product.

### Repair and Reform

If the repair and modification is necessary, please contact your dealer.

Do not modify by yourself, it may lead to accident or malfunction.

Do not replace parts or material randomly.

Do not add weight randomly, in case to cause imbalance.

Do not use other vehicle to pull or push the wheelchair when

somebody sits on the wheelchair or the wheelchair is in manual mode.

When there is abnormal sound, contact service immediately.

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Observe the conditions for storing and stowing your product:
  - In a dry and temperate place. Do not storage wheelchair in damp condition, such as near-by bathroom. If wheelchair got wet in the rain, wipe the water immediately.
  - Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
  - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
  - All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

### Protection from electromagnetic interference

The wheelchair should away from strong magnetic fields and large inductive electrical equipment, such as radio station, TV station, underground radio station, cell phone transmitting radio station. Pay attention to that if there are sources of electromagnetic interference nearby, as far as away from those sources to avoid electromagnetic interference. The electric wheelchair should avoid electromagnetic interference.

## TECHNICAL SPECIFICATIONS

Overall dimensions	Width. 67 cm x Depth. 105 cm x Height 94 cm
Seat dimensions	Width 45 cm x Depth. 42 cm x Height 43 cm
Seat Width	45 cm
Seat Depth	42 cm
Floor to seat Height	51 cm
Backrest Height	43 cm

Folded Width	67 cm
Front Caster Size	20.32 cm
Rear Caster Size	30.48 cm
Structure	Aluminium Frame
Weight	29 kg
Maximum weight supported	125 kg

<b>875025</b>	<b>SPECIFICATION</b>
Frame	Aluminum frame, powder coating black
Controller	Chinese Made controller
Motor	180W*2 LINIX CHINA with electromagnetic brake
Battery	Lithium battery 24 V, 20Ah, quick-release battery box
Charger	HP 24V 3A Taiwan
Max speed	7 KM/H
Travel mileage	10-15 KM
Gradeability	8 degree
Front wheel	20,32 PU caster
Rear wheel	30,48 solid tire, aluminum alloy wheel hub
Armrest	Flip back armrest
Footrest	Left up and down
Seat	Detachable seat cushion
Optional	Anti-tip, shopping bag

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition. User should do not overuse more than service lifetime in case of unpredictable accident.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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