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EN

OVERVIEW

This manual is only used as a reference for the operation of the product, please read and understand the user manual before using it as the manufacturer cannot assume any consequences and liability for any damage incurred by using this manual for other purposes, or if the user fails to perform maintenance activities or ignores warning messages.

Please familiarize yourself with the appliance by reading this manual before putting it into operation. If you do not understand any part of this manual or if you need additional services, please contact your local dealer authorized by our company. The following symbols are used to identify warnings and cautions. They are used throughout the user manual and are very important! It is essential that you observe the warnings and safety information specified

 WARNING	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
--	---

- This lift transfer chair is specially designed for short distance indoor movement of disabled, elderly or weak people weighing less than 150kg, such as transferring between bed, sofa, wheelchair, toilet, table and bathroom. It is an ideal moving tool for use in homes, hospitals, retirement homes and other flat roads.
- This product is removable, easy to transport and store.
- The seat frame can be quickly opened and closed, which is convenient for use in homes, hospitals, nursing homes and other flat roads. It is not recommended for use in outdoor environments such as lawns, gravel roads, slopes greater than 6° and car lanes.

The application elements of this product are the seat (integrated with the backrest and armrest) and the seat cushion, which serve to support the patient

SAFETY PRECAUTIONS

This transfer chair is an active, non-sterile, reusable product. Please read the instruction manual for the correct use of this product. If you have specific needs, please contact your local authorised dealer or service center.

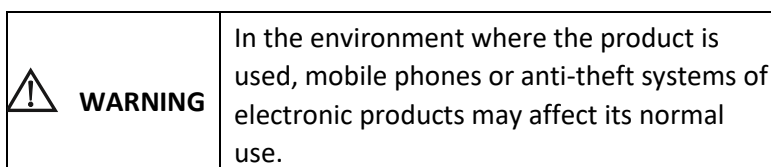
Attention to safety

- Never use a transfer chair that is not working or is damaged.
- If the transfer chair is not checked or if the castor brake fails, stop using it and contact your local dealer or service centre.
- To reduce the risk of fire or electric shock, do not place the equipment in a damp area with the battery. Do not disassemble the equipment yourself. If the appliance needs to be repaired, the service must be carried out by qualified and authorised service personnel of an organisation licensed to carry out such work.
- If the equipment emits smoke or any other abnormal condition, which may be dangerous (such as fire), please disconnect the power supply immediately and contact your local dealer or service centre.
- If the battery or electrical parts get wet, which may cause dangerous situations (such as short circuit), please remove the battery case immediately and contact your local dealer or service centre.
- Do not disassemble the equipment, no modification of this equipment is allowed. Unauthorised modifications will void the warranty.

- Before charging the equipment, please check that the power cord is not damaged. Do not use a damaged charger, and contact your local dealer or service centre.
- In case of thunder or lightning, please disconnect the power supply and do not charge.
- Do not touch the charger's power cable with wet hands to avoid the risk of electric shock.
- Do not use insecticides, alcohol, thinners and chemical liquids to clean the equipment, as this may cause the surface of the equipment to peel or rot.
- The recommended service life of the Lift Transfer Chair is five years in normal operation. Longer use may lead to ageing of the mechanism, malfunction and even accidents. Please contact a qualified medical device recycling agency for recycling and replacement of the new device.
- Do not touch the I/O ports as this may cause injury to the patient.
- This product is not suitable for use by children under the age of 12.
- Replacement should be performed by qualified professionals. Our company can provide circuit diagrams, troubleshooting guidelines, or useful maintenance materials to professional maintenance personnel.
- This product is not a toy. It should not be used by children. Be careful to prevent strangulation and suffocation caused by the cable.
- A warning that the connection of electrical equipment to the MSO does lead to the creation of a ME SYSTEM and that this may result in a reduced level of safety.

PREVENTION OF ELECTROMAGNETIC INTERFERENCE

- Do not use the handheld radio transceiver when the transfer chair is switched on, and avoid approaching radio transmission systems, such as radio stations.
- Under very extreme conditions, electromagnetic interference may occur between this device and other devices. Do not use this device with sensitive equipment or equipment that may produce strong magnetic interference.
- Caution: Mobile phones or anti-theft systems may affect their normal use.

WHAT YOU NEED TO KNOW BEFORE FIRST USE

- Please do not use the transfer chair until you have read and fully understood this manual.
- The maximum load rating for this transfer chair is 150kg, and is limited to one person, please do not overload it.
- The carer must have the physical, mental and physical strength to use the transfer chair safely. The caregiver should not be visually impaired and should be able to read and understand the manual without difficulty. Before using the wheelchair for the first time, please practice several times without a passenger.
- Do not use it in low light or with blurred vision.
- For the first time use, please refer to chapter "Installation and Troubleshooting" for the reset operation, and refer to chapter "User's Guide".
- The waterproof degree of this product is IP25, please do not expose it to humidity and shower for a long time. You can

use the product in the shower, as long as you remove the battery.

- When the temperature is below -20°C or above 55°C, do not use the transfer chair.
- If the product is removed from the -20°C storage environment, it should wait for two hours at room temperature, and can be used after drying.
- Please charge it before using it for the first time, insert the most of the charger into the charging port of the battery, and wait for the charger indicator light to turn green.
- When the ambient temperature is below 0°C, please do not charge it to avoid damaging the battery and causing injury.
- Please check the packaging for damage before unpacking, in case you need to repair or exchange it. If the original packaging is missing, returns will not be accepted.
- The caregiver must have the physical, mental and flexibility ability to operate the controller safely, not be visually impaired and be able to read the instructions correctly. When using for the first time, please practice several times without a passenger.

OPERATING INSTRUCTIONS

- The transfer chair is suitable for use in homes, hospitals, nursing homes and on other flat roads. Do not stop the chair on a slope. The transfer chair can only be moved or used by the carer, the patient sitting in the chair is not allowed to operate it.
- This transfer chair is equipped with four wheels with brakes, before moving it, please make sure that the four brakes are released, after reaching the target position, please press the brake pedals to lock the wheels.
- After inserting the battery, press the up or down button to turn on the transfer chair. If the indicator does not light up, please check if the battery is in place. When used for the first time, the transfer chair should be "re-initialised" before use. Please refer to chapter 4 "Installation and Commissioning" and follow the guide.
- Please refer to the picture guide in Chapter 5 "User's Guide" to help the patient sit or stand up from the chair.
- Before moving, please ensure that the backrest safety belt and the latch are locked, that the patient is seated properly and that the safety belt is fastened.
- When transferring the patient to the destination, the caregiver should lower the chair to an appropriate height while ensuring safety to avoid injury or property damage. When using the lowering function, please stop in advance when approaching the target object (sofa, bench, bed or toilet, etc.), and then press the lowering button briefly to adjust the height gradually. If the target object is a hard object, avoid direct contact between the transfer chair and the object (leave a 1-2 cm gap).
- The waterproof level of the transfer chair is IP25, it has anti-splash function. It can be used in the bathroom shower. You can use the product in the shower, as long as you remove the battery. The transfer chair's four-wheel brake switch must be locked before bathing to prevent slipping. This transfer chair cannot be used by immersing it in water. If the bath is not well drained and water accumulates up to the height of the battery bar, please stop the shower and immediately move the chair to a place where there is no water. You can continue to use the chair once the water has drained.
- The cushion of this chair is not waterproof, please remove it before entering the bathroom.
- If the panel is damaged, there is a risk of water infiltration, please do not use it in a bathroom, contact the dealer or manufacturer for repair.
- There is a bed tray mounting frame under the seat back for quick installation of the bed tray.
- The life span of this transfer chair in clinical use depends not only on the structural strength, but also on the conditions of use, user habits and proper maintenance.

WARNING

1. Before sitting in the transfer chair, make sure the front wheel brake is locked to prevent movement, otherwise you may tip over or fall.
2. Attach the backrest safety buckle and safety lock before moving to avoid accidents.
3. Before moving, please make sure that the backrest safety buckle and safety lock are locked and that the patient is seated properly.
4. Do not charge the battery during or after a shower while the chair is still wet.
5. Please do not use the transfer chair without an attendant, as this may cause injury.
6. The patient is "NOT" the intended operator. When the patient is sitting in the transfer chair alone, do not operate the transfer chair by itself (in case there are obstacles under the seat). Do not press the up/down button, the patient may not notice if there is an obstacle under the seat.
7. When the caregiver is using the transfer chair, do not move the chair if the lock is not locked and the patient's safety belt is not fastened, otherwise there is a risk of injury.
8. The transfer chair should not be serviced or repaired while a patient is sitting in it.
9. Do not put your hands on the quick-release folding clamp to avoid pinching.
10. Keep your feet away from the column motor to avoid injury when the chair is lowered.

Accessible materials and information on composition

Information on materials and ingredients with which patients or operators come into contact.

N°	Name of spare parts	Material composition
1	Material of the armrest and backrest surface	PU foam
2	Push handle	Rubber
3	Coussins et ceintures du siège	Seat cushions and belts
4	Control panel	ABS
5	Surface material of the safety doorway	ABS
6	Rubber lining of the bottom frame	PP

PRODUCT COMPONENTS

UNPACKING AND INSPECTION

When taking the transfer chair out of the box, please check that the following components are complete.

2-piece rear wheel
Component

Main frame (integrated
with two lift motor columns)

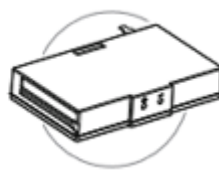
Seat component: left backrest 1 piece

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2-piece push handles
droit 1 pièce

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Safety belt 1 piece

Bedpan 1 piece

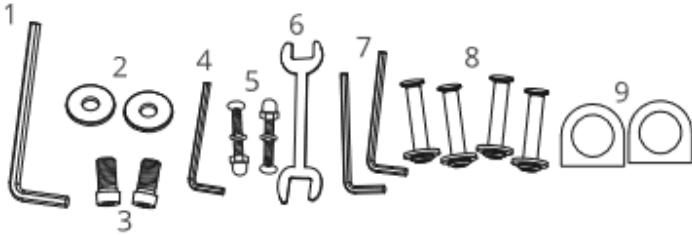
Support for bedpan 1 piece

Charger 1 piece

Seat cushion 1 piece


Installation tool:		
1.	8 mm hexagonal key	1 piece
2.	Gasket M10	2 pieces
3.	Screw M10 x 20	2 pieces
4.	4 mm hexagonal key	1 piece
5.	Screw M6x30/M6x35	1 piece
6.	No10-13 opened hexagonal key	1 piece
7.	5 mm hexagonal key	2 pieces
8.	Screw M8x40 (with spacers)	4 pieces
9.	Seat frame fixing joint	2 pieces


COMPONENT DIAGRAM

- Push handle: The handle used by the carer to push the transfer chair.
- Seat/backrest component: The user moves on the armrest and backrest.
- Seat frame: The frame can be turned and closed, which is convenient for defecation.
- Bedpan: Plug-in pendant, convenient for going to the toilet.
- Lift motor columns: Provide lifting power and support the main arms.
- Front and rear castors: equipped with brakes, they provide support and stability to the lift.

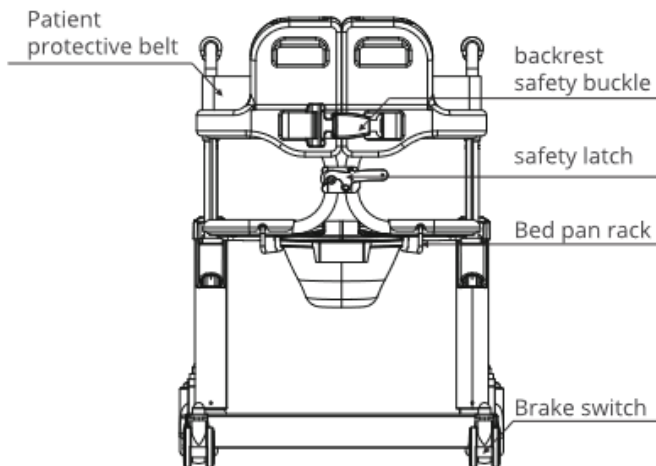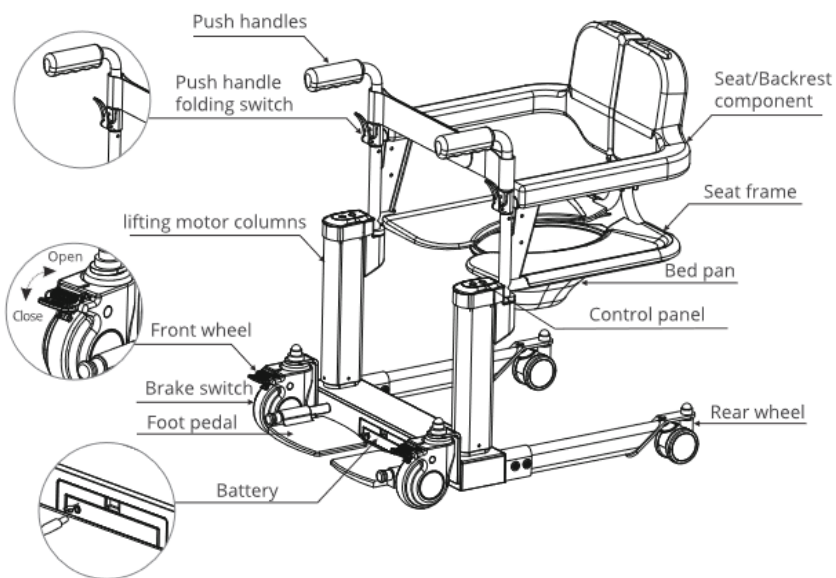The application elements of this product are the seat (integrated with the backrest and armrest) and the seat cushion which serve to support the patient.


INSTALLATION AND START-UP

INSTALLATION

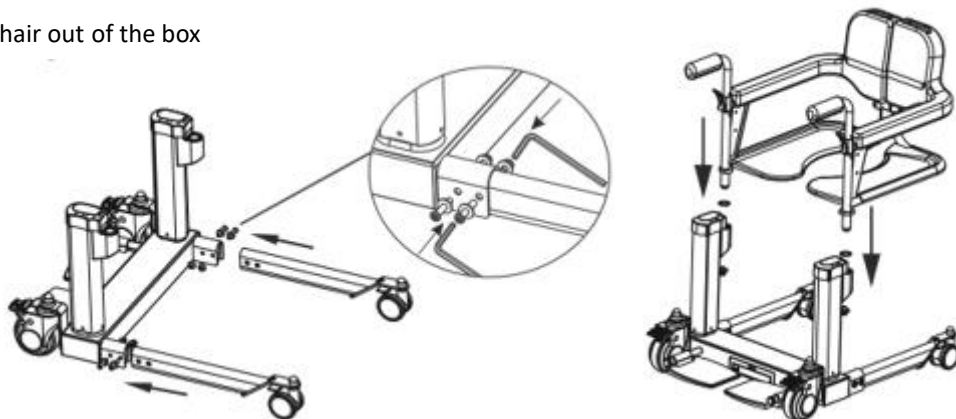1. Slide the transfer chair out of the box


2. Install the two rear wheel components onto the main frame. Use the screws that are already mounted on the main frame.

3. Insert the left and right components of the seat/backrest into the holes in the seat frame.


Secure them with the screws from below. Insert the 2 push handles into the seat/backrest components and secure them with screws.

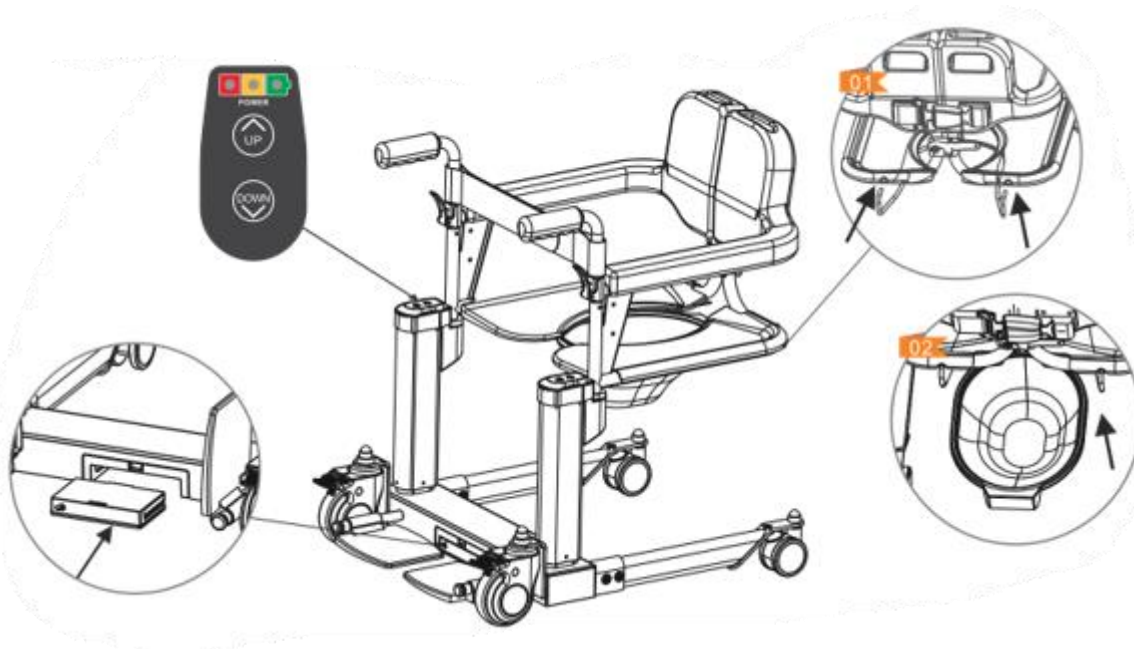In standby mode (no indicator lights are on), press the up and down buttons simultaneously for 3 seconds to force a reset.


4. After assembly, insert the battery into the slot between the two front wheels.
Press any button on the panel to start the transfer chair. Before first use, please refer to the commissioning guide and perform a reset.
Before using for the first time, please refer to the commissioning guide and perform a reset.
5. Install the accessories:
Patient protection belt; bed tray; bed tray support; seat cushion; battery.

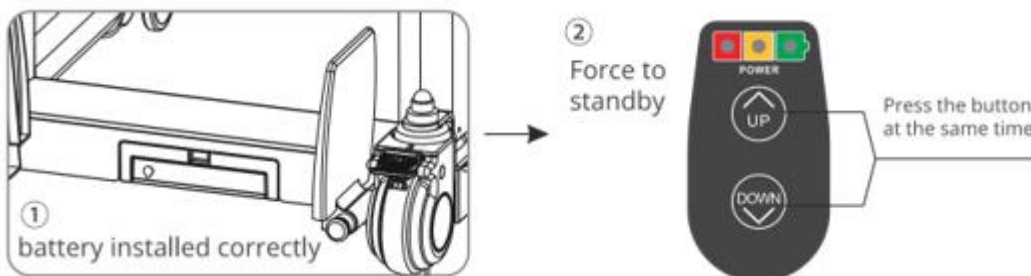PUTTING INTO SERVICE


1. Before use, ensure that the battery is correctly installed and that the transfer chair is powered on.
2. For the first time use, a reset operation is required. Before use, put the chair into standby mode in one of two ways:
 - a. The chair will automatically go into sleep mode after 1 minute of inactivity.
 - b. Or press and hold the up and down buttons simultaneously for 3 seconds to force the chair to sleep mode.
3. Reset: In standby mode (no indicator light is on), press and hold the up and down buttons simultaneously for 3 seconds to force the reset. Please ensure that no one is moving during the force reset. While the force reset is in progress, if there is an accident, press any button to stop the reset.
4. During the hard reset, if an overload or voltage loss occurs, the circuit has a protective function, and the column lift will stop immediately.
5. When there is uncoordinated unilateral lifting, it is necessary to carry out inspections in unloaded conditions to find the abnormal points.
6. If an uncontrollable abnormality occurs, quickly remove the battery.

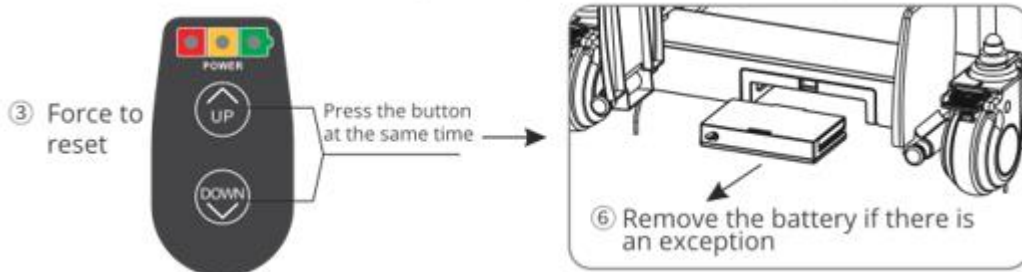USER GUIDE

CONTROL PANEL

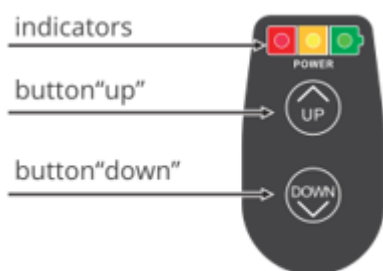
It is located at the top of the left lift motor column. There are three indicators, red, yellow and green, from left to right.

Battery indicators :

1. Flashing red light, low battery, stop using the chair, and recharge in time.
2. Flashing yellow light, 15-50% power left, pay attention to the battery level and charge it if necessary.
3. Green light on, the battery is more than 50% charged, no charging is needed.

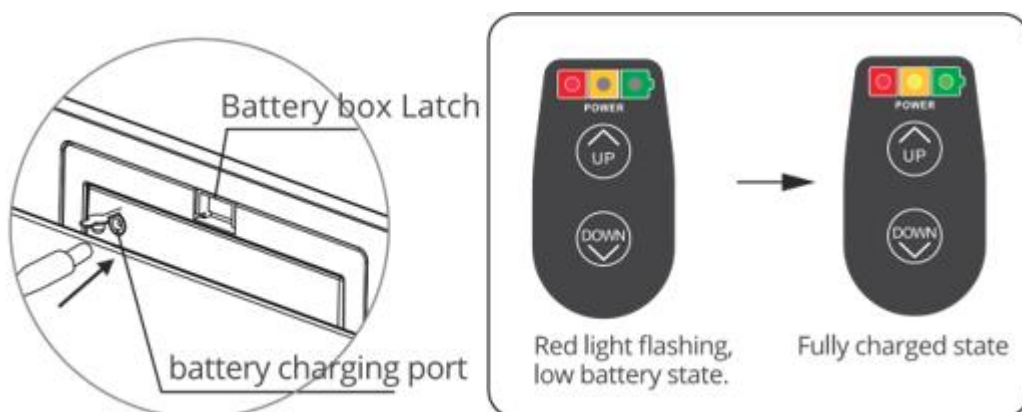LOADING OPERATION

- Insert the tips of the charger into the charging port of the battery and plug in the power supply to the charger. (The battery can be charged separately or directly on the wheelchair).
- When charging, the charger light is red, and turns green when the battery is fully charged, then remove the charger.
- If the battery overheats or smokes during charging, stop charging immediately and move the battery to a safe place quickly.
- Overcharge protection: If the ignition engine does not work properly, an obstacle may be causing the overcharge, remove the obstacle and press the button again.

1. If you connect equipment other than the original chargers to the battery charging port and damage the control system, the warranty will naturally become null and void.

WARNING

2. When the ambient temperature is below -20°C or above 55°C, please do not use the transfer chair. Do not use the transfer chair.
3. The original charger is a 29.4V/1.2A charger, which complies with part 1 of the EN60601-1 Medical Electrical Equipment: General Safety Requirements.
Chargers are not permitted.
4. When the ambient temperature is below 0°C, please do not charge it to avoid damage to the battery to avoid damage to the battery and injury.
5. Do not place the equipment in such a way as to make it difficult to use the power to electrically isolate the equipment circuits from the mains supply. power supply

TRANSFER

1. Move the transfer chair close to where the patient is sitting or lying in bed.
2. Unlock the buckle and latch on the back of the backrest to separate it.
3. By pressing the up/down button, adjust the seat to the appropriate height (2-3 cm higher than the chair, sofa or bed the patient is sitting on).
4. Push the transfer chair inwards, lock the front wheel brake switch so that the lift cannot move. Let the patient step on the chair's foot pedal.
5. Help the patient to sit in the chair by closing the left and right backrests.
6. When the patient is seated properly and the left and right backrests are closed, lock the safety buckle and the latch. The chair can now be moved.

TROUBLESHOOTING

This product has an abnormal warning function. When a failure occurs, the three LED indicators on the control panel flash at the same time. Please check according to the warning signal table. If the problem persists after the check, please contact your service agent.

Anomaly analysis table		
Anomaly	Analysis	Troubleshooting
All three indicators are flashing	External force or contact with other objects during the reset process, causing anomalies.	Remove obstacles and perform the reset operation again. In standby mode (all three lights are off), press the (up) and (down) buttons simultaneously for 3 seconds to initiate the reset function.

SAFETY PROTECTION AND ACCIDENT HANDLING

Loading: See Chapter "User's guide -- Loading operation".

If the transfer chair fails during lifting, if it cannot be stopped when the button is released or a button is pressed, please remove the battery immediately to stop the lift forcefully.

MAINTENANCE AND DISINFECTION

This transfer chair requires regular care and maintenance. The user can carry out a routine check and maintenance, other services should be carried out by the dealer. Proper maintenance can keep the transfer chair in good working order. If you encounter any problems during maintenance, please contact your dealer or the service department.

Pay attention to the following points during maintenance:

- **Moisture:** Avoid placing the transfer chair in a damp environment for long-term storage. Soaking in water or storing in a damp environment will cause mechanical parts to rust easily and electronic components to corrode.
- **Temperature:** In extremely low temperatures, the battery may freeze. Temperatures above 55°C may cause the lift controller to overheat.
- **Routine check :**
 - a) Regularly check the connection between the battery and the control box to ensure that they are in good contact, well connected, and secure.
 - b) When the battery charge indicator is red, it means that the battery is very low and needs to be recharged immediately. Try to avoid recharging when the battery is depleted. If the wheelchair is to be stored for a long time, it should be charged at least once every 3 months.
 - c) Regularly check that the nuts securing the four brake wheels, lifting devices, joints, etc. are not loose. If they are, tighten them in time.
 - d) Regularly check that the battery bracket is completely sealed and that the control panel, buttons, and plastic parts are intact. If cracks appear, stop using it and contact the dealer or manufacturer immediately.
- **Routine care**
 - a) Avoid hitting or knocking the lifting column, especially the control panel.
 - b) Avoid prolonged exposure to harsh conditions, such as excessive heat, cold and humidity.
 - c) Keep the control panel clean.
 - d) The surface of the frame of this product is coated with a smooth sealing layer. Users can use car wax to keep the surface shiny.
 - e) The battery contacts should be kept clean and well connected to the contact pins. They should not be dirty or loose.
 - f) The wheel bearing has its own lubricant and has been sealed, so there is no need for the user to add lubricant.
- **Disinfection**

Disinfection with alcohol should be carried out after use by different patients. Use a clean cloth or sponge containing 75% alcohol to gently wipe the material that patients may touch, without wiping vigorously in the same position. (Note: During use, be careful not to spill high concentrations of alcohol, and the concentration of alcohol in the air should not exceed 3% to avoid fire)
- **Daily care after use :**
 - a) If not used for a long time, please remove the battery.
 - b) When cleaning the lift, wipe it with a clean, soft cloth. Do not use chemical cleaners to avoid discoloration and deformation of structural parts.
 - c) After using the bed tray, remove it to clean it in time. The seat frame can be cleaned with clean water or wiped with a towel.
 - d) The cushion cover can be removed and cleaned and can be used after drying.
 - e) The bedpan should only be hooked to the seat frame when necessary, so as not to affect the opening and closing of the backrest/seat.
 - f) After showering with the chair, wipe the surface with a soft cloth as soon as possible and place it in a ventilated area to keep the lift dry.

Attention!

When the product is exposed to external heat sources (such as sunlight), the surface temperature increases, which affects the use and safety of the product.

- **Control list :**

To keep the transfer chair in good condition, check before each use, and check regularly as follows:

Check the item	Any time	Weekly	Monthly	Yearly
Four wheels	X			
Belt of protection	X			
Battery level	X			
Control panel function	X			
Cleanliness	X			

Screws and fasteners		X		
Lifting motor column			X	
Frame stability			X	
State of the seat				X
Brakes			X	

- **Transport**

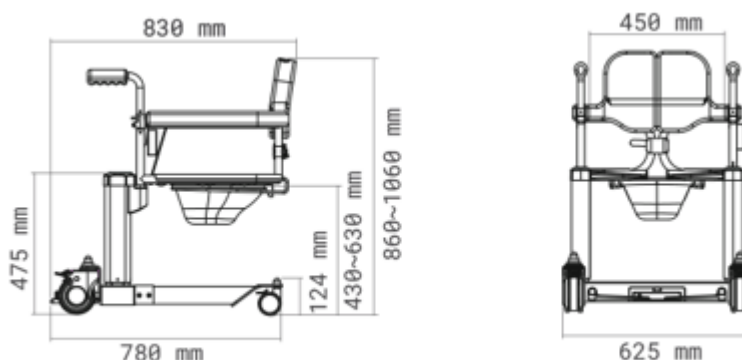
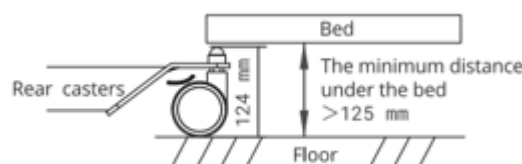
Load and transport in accordance with the signs and transport graphics on the packing carton. For more details, see Chapter 10 "Names of marks and graphic symbols".

- **Storage**

The packed transfer chair should be stored at a temperature of -20°C to +55°C, with a relative humidity of 0% to 80%, free of corrosive gases and in a well-ventilated room. When storing, remove the battery. If it cannot be stored in the above conditions, the holder may rust, and the electrical system may be damaged.

TECHNICAL CHARACTERISTICS

- Mobility type : indoor type
- Waterproofing level: IP25
- Dimensions (mm): 830mm x 620m x 860mm
- Detachable
- Seat cushion dimensions : 450mm x 450mm x 70mm. Seat cushion area on floor: 520mm
- Net weight: 36kg, Gross weight: 38kg
- The distance from the bottom of the seat frame to the floor is 440mm, and the distance from the highest point of the rear castors to the floor is 125mm. Note: The minimum distance between the rear wheels of the transfer chair and the bottom of the bed is greater than 125mm.
- Loading capacity: 150 kg
- Lifting speed of motor column: loaded approx. 8mm/s, unloaded approx. 10mm/s
- Lifting stroke: 200mm
- Radius of turning circle: 950mm
- Front wheel size: 5
- Rear wheel size: 3'
- Motor: 24V/100W x 2pcs
- Li-on battery: 25.9V/2600mAh
- Environmental conditions of use: 0°C - +40°C <80% R.H. 70kPa to 106kPa
- Conditions de stockage et de transport : -20°C à 55°C ; <80% H.R. 70kPa à 106kP

WARRANTY

Three-year warranty for the main frame. One year warranty for the following electronic components:

- Controller, control panel, motor, charger, battery.
- Within one year from the date of purchase, the service department has verified and confirmed that it is a production defect, and the customer is the original purchaser.
- Wearing parts are not covered by the warranty: Plastic shell, armrest, backrest, rubber cushion, castors, decorative parts, seat back cushion.

The following problems are not covered by the warranty:

- Damage caused by abuse and improper use.
- Damage caused by incorrect maintenance and storage
- Damage caused by commercial or other abnormal use.
- For service, please refer to the electrical nameplate and body code information of this product and provide it to the service department.

Attention!

Used batteries and other casts must be recycled in accordance with the user's local regulations and may not be discarded at will.

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